**HUMAN CARE**  
Your Life. Your Way.

## Rollator Carl-Oskar / Rebel

Bruksanvisning/User Manual/Brukermanual/  
Brugermanual/Ohjekirja/ Gebruikershandleiding

## Innehåll

SVENSKA

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# User Manual

## Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

## Symbols

**WARNING** symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

**INFORMATION** symbol indicates recommendations and information for proper, trouble-free usage.

**WEIGHT CAPACITY** symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

**CE MARKING** symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**MEDICAL DEVICE** symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**RECYCLE** according to local regulations.

**READ INSTRUCTIONS** and ensure you fully understand them before using this product.

***Product modification is not permitted.***  
***Before use, always check that the product is not damaged or worn.***

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at [www.humancaregroup.com](http://www.humancaregroup.com).

Contact your local distributor if you have any questions about the product and its use.

### Notice to user/patient in case of serious incident

Any serious incident that has occurred in relation to the product, should be reported to the local contact, who reports to the manufacturer, and the competent authority of the country in which the user/patient is established.

## Compliances and Standards

---

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This product has been tested in accordance with ISO 11199-2:2021.

## About our products

---

### On delivery

Check the following before using your new product:

1. Check that no damage has occurred during transport (if this is the case, please immediately inform the delivery agent).
2. Check that the number and type (s) of products detailed on the delivery certificate agree with the goods delivered.
3. Check that the delivery is of the desired model and type of product.

Please contact us immediately in case of any delivery problem.

### Return or refund

If any product has to be returned for any reason, please contact us to obtain a claim number and advice on how to proceed.

The goods should be returned in complete and undamaged condition in their original packaging.

The following information should accompany any product on its return. The product's identification number, delivery date, the date the user took possession of the aid, the date of the product's last service, the person whom we should contact about the fault, whether it be the user or another, and the nature of the fault.

Human Care HC AB reserves the right to inspect the returned product. If no fault can be found, the item will be returned to the user. Transport costs may then be chargeable.


### Product lifetime

The product's expected lifetime is seven (7) years, if the product is used as intended and maintained according to the manufacturer's instructions, depending on the intensity of use and maximum load applied during use. If the product label is no longer legible, the product should be discarded.

# Product label

The product label contains information about the rollator.

The label is located on the right lower frame tube, close to the right rear wheel.

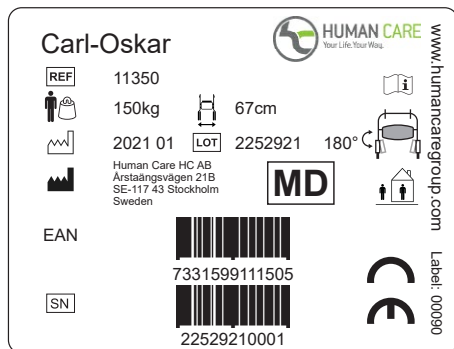	Article number
	Maximum user weight
	Date of manufacture
	Address of manufacture
EAN	UPC code
	CODE 39 barcode, Serial number

	Read the manual
	Product width
	Indoor / outdoor use
	CE marking
	MD marking

## Additional Warnings

Ensure that all bolts are securely fastened before using the rollator!

DO NOT adjust brake tension until wheels lose molding sheen!

Brakes must be in the parked/locked position before sitting on the rollator!

DO NOT hang anything on backstrap or it may break!

Ensure non entrapment of fingers on all moving parts.

# Rollator Carl-Oskar / Rebel

## Safety

In the instruction manual, three ways of drawing special attention are used. These ways are highlighted in the following manner.

### Attention!

Risk of personal injury or mechanical fault.

### Important!

Risk of mechanical damage or fault.

### Note!

Important information to ease handling.

## Recycling of products

When recycling, the product's chassis is separated from other parts. At separation, the parts can be sorted according to material after unscrewing and dismantling rivets and other connections. The different parts should be separated into metal, plastic or combined materials.

The products consist mainly of steel pipe and other metal components.

Examples of parts which consist of combined materials, are the plastic front forks (built up in several layers), wheels and other components, which consist of a composite of materials.

### Note!

Different authorities/recycling companies handle recycled materials in different ways. It is thus fitting when in any doubt over the best way to proceed when recycling, to contact the relevant governing body.

## Environment friendliness

Human Care products are manufactured so as to have as little effect on the environment as possible during their lifecycles. This allows for the expired product to be dismantled and for its parts to be recycled. Products with lengthy lifecycles safeguard our sensitive environment and are thus more environmentally friendly.

Human Care HC AB is a part of FTI. FTI – A countywide system for recycling of packaging.

## Intended use

Carl-Oskar and Rebel are stable rollators, suitable for use both out- and indoors. With their flexible chassis, they work well even on uneven surfaces. Both rollators have a unique way of folding, enabling them to be stored upright whilst folded. They can be folded and unfolded in one simple operation. Carl-Oskar and Rebel are available as reinforced models for users weighing up to 275 kg.

## Technical Data

Folded

Max user weight

Rollator Art nr.*	Min	Max	Seat height	Width	Seat width	Length	Wheel size	Weight (kg)	LxWxH (Min.)	Standard (kg)
Carl-Oskar										
11150	770	950	640	570	370	820	300x45	11,1	820x260x770	150
11250	770	950	640	620	420	820	300x45	11,3	820x260x770	150
11350	770	950	640	670	470	820	300x45	11,4	820x260x770	150
11450	770	950	640	720	520	820	300x45	11,5	820x260x770	150
Carl-Oskar High										
11154	820	1000	640	570	370	820	300x45	11,1	820x260x770	150
11254	820	1000	640	620	420	820	300x45	11,3	820x260x770	150
11354	840	1020	640	670	470	820	300x45	11,4	820x260x770	150
11454	840	1020	640	720	520	820	300x45	11,5	820x260x770	150
Carl-Oskar air pumped wheels										
11151	770	950	640	570	370	820	300x45	11,1	820x260x770	150
11251	770	950	640	620	420	820	300x45	11,3	820x260x770	150
11351	770	950	640	670	470	820	300x45	11,4	820x260x770	150
11451	770	950	640	720	520	820	300x45	11,5	820x260x770	150
Carl-Oskar reinforced										
61225	770	950	640	620	420	820	300x45	13,2	820x260x770	275
61325	770	950	640	670	470	820	300x45	13,3	820x260x770	275
61425	770	950	640	720	520	820	300x45	13,4	820x260x770	275

(All measurements in mm)

\* When ordering, make an addition after the part number for the package size, QX, where Q is for quality and X for the size of the package. Please see [www.humancaregroup.com](http://www.humancaregroup.com) for more information.

# Rollator Carl-Oskar / Rebel

Folded

Max user weight

Rollator Art nr.*	Min	Max	Seat height	Width	Seat width	Length	Wheel size	Weight (kg)	LxWxH (Min.)	Standard (kg)
<b>Rebel</b>										
22140	750	930	620	570	370	750	200x40	9,5	750x250x740	140
22240	750	930	620	620	420	750	200x40	9,6	750x250x740	140
22340	750	930	620	670	470	750	200x40	9,7	750x250x740	140
22440	750	930	620	720	520	750	200x40	9,9	750x250x740	140
<b>Rebel Low</b>										
22141	680	860	540	570	370	750	200x40	9,2	750x250x680	140
22241	680	860	540	620	420	750	200x40	9,4	750x250x680	140
22341	680	860	540	670	470	750	200x40	9,5	750x250x680	140
22441	680	860	540	720	520	750	200x40	9,7	750x250x680	140
<b>Rebel High</b>										
22144	800	980	620	570	370	750	200x40	9,2	750x250x680	140
22244	800	980	620	620	420	750	200x40	9,4	750x250x680	140
22344	820	1000	620	670	470	750	200x40	9,5	750x250x680	140
22444	820	1000	620	720	520	750	200x40	9,7	750x250x680	140
<b>Rebel reinforced</b>										
71221	750	930	620	620	420	750	200x40	11,5	750x250x740	250
71321	750	930	620	670	470	750	200x40	11,6	750x250x740	250
71421	750	930	620	720	520	750	200x40	11,8	750x250x740	250

(All measurements in mm)

\* When ordering, make an addition after the part number for the package size, QX, where Q is for quality and X for the size of the package. Please see [www.humancaregroup.com](http://www.humancaregroup.com) for more information.

## Accessories

Accessories	Art. no.	
Handle Cobra H	00022	
Handle Cobra V	00021	
Basket with handles	10601	Important! Max load 10 kg
Tray for Carl-Oskar and Rebel	10651	Max load 2 kg
Cane holder	10603	
Back support for 57, 62, 67	10621	
Back support for 72	10655	
Oxygen holder 1-5 l	10615	Max load 5 kg
Slow down brake	10624	
One-hand brake	10652	
Curb climber (Not for reinforced models)	10612	
Splash guard (For Carl-Oskar)	10608	
Cross handle	70844	

See respective assembly instruction.

Table 1. Possible accessory combinations

	Basket	Tray	Cane holder	Back support	Oxygen holder	Slow down brake	One-hand brake	Curb climber	Splash guard	Cross handle
Basket	✘	✓	✓	✓	✓	✓	✓	✓	✓	✓
Tray	✓	✘	✓	✓	✓	✓	✓	✓	✓	✓
Cane holder	✓	✓	✘	✓	✓	✓	✓	✓	✓	✓
Back support	✓	✓	✓	✘	✓	✓	✓	✓	✓	✘
Oxygen holder	✓	✓	✓	✓	✘	✓	✓	✓	✓	✓
Slow down brake	✓	✓	✓	✓	✓	✘	✓	✓	✘	✓
One-hand brake	✓	✓	✓	✓	✓	✓	✘	✓	✓	✘
Curb climber	✓	✓	✓	✓	✓	✓	✓	✘	✘	✓
Splash guard	✓	✓	✓	✓	✓	✘	✓	✘	✘	✓
Cross handle	✓	✓	✓	✘	✓	✓	✘	✓	✓	✘

✘ = Not applicable.




# Rollator Carl-Oskar / Rebel

## User guide

- Read all warnings and instructions carefully before using the rollator.
- Ensure that the rollator is properly unfolded before use.
- The rollator may only be used as a support for walking or sitting.
- Never stand on the rollator.
- Do not use the product on stairs or escalators.
- Be extra careful when using the rollator on slopes or uneven surfaces.
- Never sit on the rollator and roll yourself forward with your feet.
- Distribute weight evenly to prevent the rollator from tipping when you sit on it.
- Always set the parking brake before using the rollator for support while sitting down or getting up.
- Do not place anything on a folded up rollator when it is in storage or being transported as this may cause damage to it.
- Do not store the rollator close to a heat source as this could cause damage to some details.
- Be careful when you handle the rollator if it has been exposed to abnormally high or low temperatures to avoid injuries.
- Do not store the rollator outdoors.
- Save the user instructions for future use.


### Unfolding

1. Detach the larger plastic part of the lock snap, and move the rollator's handle outwards.
2. Push seat forward.
3. Important! Check that the lock grips the gate.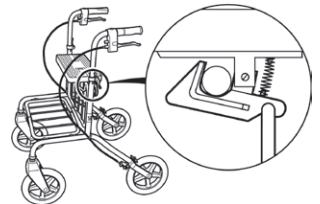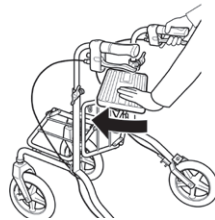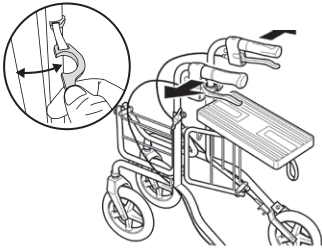


### Folding

1. Pull the red loop until the lock frees the gate.
2. Pull seat backwards and move handle inwards.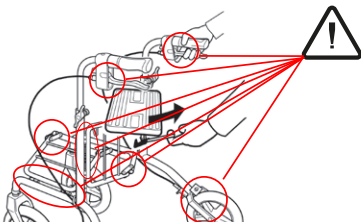


**Attention!**  
Risk for finger trap!

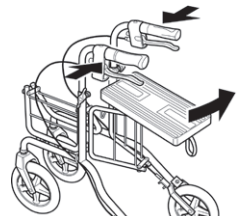
### Adjustment of handles

Loosen wingnut and adjust handgrip to wrist height. Lock wingnut.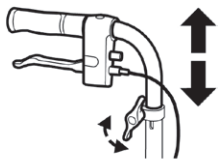

On the reinforced model, the strengthening bar should also be adjusted.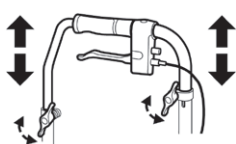

### Parking brake

Pull brake lever downwards when parked.


**Warning!**  
Engage parking brake before sitting.


### Important!

The handgrips should not be heightened above the "0" mark on the sides of the handle pipes. The handles must be faced backwards. The mark must be aligned with the stroke on the clamp at the side of the pipes to avoid instability and dangerous situations.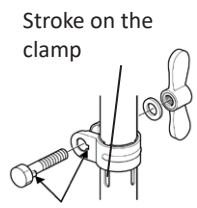

### Warning!

The handles must not be pointing outwards because of increased tip risk. The angle between the lengthwise line of the handles and the forward direction of the rollator must always be 180°.

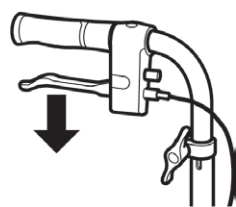
### Brake

Pull brake lever upwards to brake.


**Note!**  
Continuous braking leads to excessive wear of wheels and brakes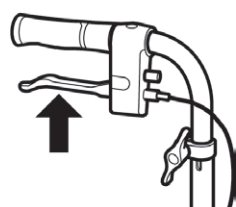

### Brake adjustment

Loosen locking nut; adjust brake by loosening/tightening the adjustment casing. After adjustment tighten locking nut.

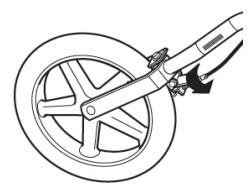# Rollator Carl-Oskar / Rebel

## Cleaning and service

### Important!

When changing parts of the rollator, use only Human Care parts to ensure continued CE validation.

### Important!

Do not alter the rollator yourself.

### Important!

Check the rollator regularly to ensure its continued safety to the user.

Regular checks should be carried out as recommended by the instructions detailed in:

Carl-Oskar, art. no.: 99211

Rebel, art. no.: 99212.

For more information on servicing, reconditioning, spare parts and accessories, see Human Care's website: [www.humancaregroup.com](http://www.humancaregroup.com) or contact us at the address, phonenumber or e-mail address according to back page.

## Warranty Terms

---

The Human Care HC AB warranty covers defects in material and manufacture of the walking aid. Wear on wheels, brake parts and handles are not included. Finish damages which have occurred after delivery are not covered. The warranty presumes that the product has been used in the intended way and that it has been regularly reconditioned in accordance with our reconditioning instructions. The ID number of the product must also be identifiable. Furthermore, the product must not have been modified or customized by any party other than Human Care HC AB. The warranty period is one (1) year or according to agreement.

For complaints, please contact us for instructions on how to proceed.

For complaints, the following information should be indicated: product ID number; the date of the last reconditioning; the name of the consignor contact; a description of the fault.

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The product label for the Carl-Oskar rollator includes the following information:

- Carl-Oskar** (Brand name)
- HUMAN CARE** logo with tagline "Your Life. Your Way."
- REF 11350** (Reference number)
- 150kg** (Maximum user weight)
- 67cm** (Product width)
- 2021 01** (Production date)
- LOT 2252921** (Lot number)
- 180°** (Adjustable seat angle)
- Human Care HC AB, Årstaängsvägen 21B, SE-117 43 Stockholm, Sweden (Manufacturer address)
- MD** (Medical Device marking)
- EAN** (UPC code) **7331599111505**
- SN** (Serial number) **22529210001**
- www.humancaregroup.com (Website)
- Label: 00090 (Label ID)

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