**HUMAN CARE**  
Your Life. Your Way.

**Trolley walker**

**RA-rollator**

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---

### Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

## Symbols

**WARNING** symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

**INFORMATION** symbol indicates recommendations and information for proper, trouble-free usage.

**WEIGHT CAPACITY** symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

**CE MARKING** symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**MEDICAL DEVICE** symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**RECYCLE** according to local regulations.

**READ INSTRUCTIONS** and ensure you fully understand them before using this product.

***Product modification is not permitted.***  
***Before use, always check that the sling is not damaged or worn.***

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at [www.humancaregroup.com](http://www.humancaregroup.com).

Contact your local distributor if you have any questions about the product and its use.

### Notice to user/patient in case of serious incident

Any serious incident that has occurred in relation to the product, should be reported to the local contact, who reports to the manufacturer, and the competent authority of the country in which the user/patient is established.

## Compliances and Standards

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This product has been tested in accordance with ISO 11199-3:2005.

## About our products

### On delivery

Check the following before using your new product:

1. Check that no damage has occurred during transport (if this is the case, please immediately inform the delivery agent).
2. Check that the number and type (s) of products detailed on the delivery certificate agree with the goods delivered.
3. Check that the delivery is of the desired model and type of product.

Please contact us immediately in case of any delivery problem.

### Return or refund

If any product has to be returned for any reason, please contact us to obtain a claim number and advice on how to proceed.

The goods should be returned in complete and undamaged condition in their original packaging.

The following information should accompany any product on its return. The product's identification number, delivery date, the date the user took possession of the aid, the date of the product's last service, the person whom we should contact about the fault, whether it be the user or another, and the nature of the fault.

Human Care HC AB reserves the right to inspect the returned product. If no fault can be found, the item will be returned to the user. Transport costs may then be chargeable.

### Product lifetime

The product's expected lifetime is seven (7) years, if the product is used as intended and maintained according to the manufacturer's instructions, depending on the intensity of use and maximum load applied during use. If the product label is no longer legible, the product should be discarded.

# RA-rollator / Gangbord Staffan

## Staffan RA

**HUMAN CARE**  
Your Life. Your Way.

[www.humancaregroup.com](http://www.humancaregroup.com)

Label: 00090

	42231		
	125kg		59cm
	2021 01		2252921
	Human Care HC AB Arstaängsvägen 21B SE-117 43 Stockholm Sweden		
EAN	 7331599422311		
	 22529210001		 

	Article number
	Maximum user weight
	Date of manufacture
	Address of manufacture
<b>EAN</b>	UPC code
	CODE 39 barcode, Serial number

	Read the manual
	Product width
	Indoor / outdoor use
	CE marking
	MD marking

## Additional Warnings

Ensure that all bolts are securely fastened before using the rollator!

DO NOT adjust brake tension until wheels lose molding sheen!

Brakes must be in the parked/locked position before sitting on the rollator!

DO NOT hang anything on backstrap or it may break!

## Safety

There are three grades of importance attached to the information in the user manual.

### Warning!

Risk for personal injury or serious malfunction.

### Important!

Risk for damage to material or malfunction.

### Note!

Important information that facilitates work procedures or handling.

## Recycling of products for disposal

When the product is to be disposed of (recycled), separate the frame from the other parts. This entails removing screws etc. and drilling out rivets, and then sorting on the basis of material - metal, plastic or mixed.

The products are mainly comprised of steel tubing and other metal parts.

Examples of parts that comprise of a mixture of materials are plastic front forks (with bearings), wheels with bearings.

### Note!

Since different municipalities/recycling companies handle recycling material in different ways, it is best, if in doubt, to contact your municipality.

## Environmentally friendly

Human Care products are produced using methods aimed at impacting the environment as little as possible, throughout their life cycle. This means that when a product is disposed of, its parts can be recycled. Products with a long service life are beneficial to our fragile environment and equate to 'environment in mind' products

Human Care HC AB is a member of REPA, a nationwide system for the recycling of packaging.

# RA-rollator / Gangbord Staffan

## Areas of use

Staffan trolley walkers and RA-rollators are stable aids that are suitable for use both indoors and out.

The trolley walker and RA-rollator are available in two models, low and high.

## Technical data

Max weight of user

Rollator Art no.*	Min.	Max.	Width	Length	Wheel size	Weight (kg)	Standard (kg)
Staffan Trolley walker							
43211 Low	860	1080	620	710	200x40	11,7	90
43221 High	1070	1280	620	710	200x40	11,9	125
Staffan RA-rollator							
42221 Low	860	1080	590	710	200x40	10,3	90
42231 High	1070	1280	590	710	200x40	10,6	125

(All dimensions in mm)

\* When ordering, make an addition after the part number for the package size, QX, where Q is for quantity and X for the size of the package. Please see [www.humancare.com](http://www.humancare.com) for more information.

## Accessories

Table 1. Permitted accessory combinations

Accessories	Art. nr. - Low	Art. nr. - High
Basket with handle*	40601	40601
Walking stick holder	40603	40603
Oxygen holder	40605	40605
Single-handed brakes	10653	10653

	Basket	Walking stick holder	Oxygen holder	Single-handed brakes
Basket	✓	✗	✓	✗
Walking stick holder	✗	✓	✓	✗
Syrgashällare	✓	✗	✗	✗
Single-handed brakes	✗	✗	✗	✓

\* Important! Max load 10 kg  
See respective assembly instruction.


✗ = Not applicable.

# User information trolley walker

## Unfolding


1. Move the table aside and unfold the frame.
2. Push the side tubes out sideways a little and then unfold them.
3. Important! Make sure the latches properly attach the side tubes to the frame.
4. Ensure that the handles are in minimum position; then fit the table and draw out the wing nuts.
5. Adjust the handles to suitable positions.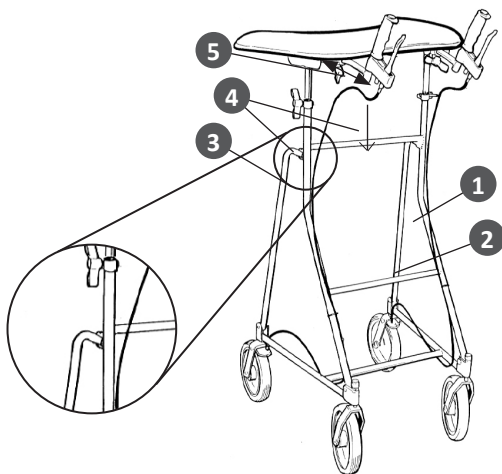

## Folding up

Before folding up the trolley, place the handles in minimum position. This is to reduce wear on the brake cables.

The trolley walker is folded up in exactly the opposite way to which it is unfolded. Place the table on top when the trolley has been folded up as shown in the picture.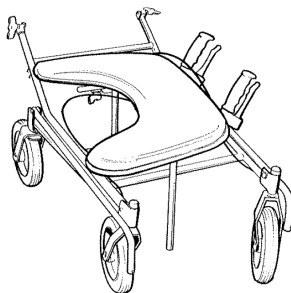


### Warning!

Risk for pinching in connection with folding up

## Height adjustment

1. Adjust the height of the support table as required by loosening the wing nuts.
2. Adjust the handles to the required positions by loosening the wing nuts under the arm supports.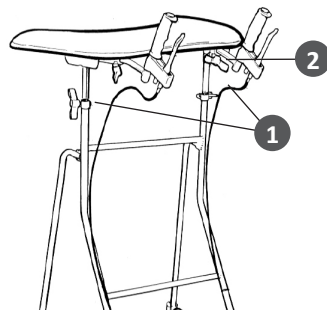

### Important!


Do not raise the support table above the "MAX" marking on the sides of the handle tubes.

## User information RA-rollator

### Unfolding

Make sure the handles are in minimum position before unfolding.

1. Push the side tubes out sideways a little and then unfold them.
2. Important! Make sure the latches properly attach the side tubes to the frame.
3. Adjust the handles to suitable positions.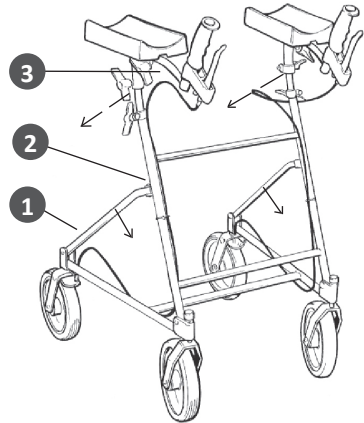

### Folding up


Before folding up the trolley, place the handles in minimum position. This is to reduce wear to the brake cables.

The RA-rollator is folded up in exactly the opposite way to which it is unfolded.

#### Warning!

When folding up, beware of the risk of pinching.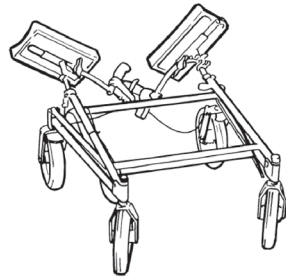


### Height adjustment

1. Set the RA supports at the required height by loosening the wing nuts.
2. Adjust the handles to suitable positions by loosening the wing nuts under the arm supports.

#### Important!

Do not raise the RA supports above the "MAX" marking on the sides of the handle tubes.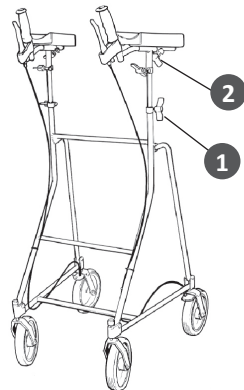


## User information trolley walker & RA-rollator

- Read all warnings and instructions carefully before using the RA-rollator/TW.
- Ensure that the RA-rollator/TW is properly unfolded before use.
- The RA-rollator/TW may only be used as a support for walking.
- Never stand on the RA-rollator/TW.
- Do not use the product on stairs or escalators.
- Be extra careful when using the RA-rollator/TW on slopes or uneven surfaces.
- Never sit on the RA-rollator/TW and roll yourself forward with your feet.
- Distribute weight evenly to prevent the RA-rollator/TW from tipping when you sit on it.
- Always set the parking brake before using the RA-rollator/TW for support while sitting down or getting up.
- Do not place anything on a folded up RA-rollator/TW when it is in storage or being transported as this may cause damage to it.
- Do not store the RA-rollator/TW close to a heat source as this could cause damage to some details.
- Be careful when you handle the RA-rollator/TW if it has been exposed to abnormally high or low temperatures to avoid injuries.
- Do not store the RA-rollator/TW outdoors.
- Save the user instructions for future use.


### Parking brake

Push brake handle downwards to park.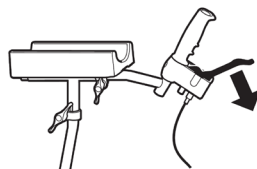

### Brake

Pull brake-handle inwards to brake.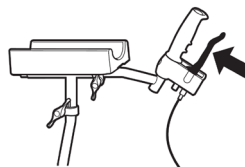


#### NOTE!

Constant braking leads to excessive wearing of wheels and brakes.

### Brake adjustment

If the brakes are not working properly, adjust the brake cable by screwing outwards.

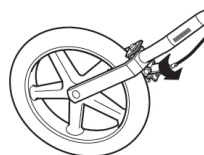# RA-rollator / Gangbord Staffan

## Cleaning and service

### Important!

When changing parts of the rollator, use only Human Care parts to ensure continued CE validation.

### Important!

Do not alter the rollator yourself.

### Important!

Check the rollator regularly to ensure its continued safety to the user.

Regular checks should be carried out as recommended by the instructions detailed in:

Staffan GB, art. no: 99217

Staffan RA, art. no: 99218

For more information on servicing, reconditioning , spare parts and accessories, see Human Care's website: [www.humancaregroup.com](http://www.humancaregroup.com) or contact us at the address, phonenumber or e-mail address according to back page.

## Warranty

---

Human Care HC AB has high demands on our product quality and our warranty cover material and manufacturing faults of our products. The warranty is not covering wear and tear on wheel, brakes, handles grips and surface damages arisen after delivery. The warranty is valid if the product is used in the intended way and the maintenance instructions have been followed. The product identity needs to be readable. The product shall not have been modified or changed in any way after the delivery from Human Care. The warranty time is 1 year if noting else has been stated in contract.

In case of product complaint please contact us for further instructions.

The following information is needed to create a product complaint:

Id number of the product, date for when the last maintenance were performed, customer contact person and failure description.

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