**HUMAN CARE**  
Your Life. Your Way.

## Fellow Supreme

85400

Bruksanvisning/User Manual/Brugermanual/  
Gebrauchsanweisung/Notice d'utilisation

## Innehåll

SVENSKA

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## Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

## Symbols

**WARNING** symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

**INFORMATION** symbol indicates recommendations and information for proper, trouble-free usage.

**WEIGHT CAPACITY** symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

**CE MARKING** symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**MEDICAL DEVICE** symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**RECYCLE** according to local regulations.

**READ INSTRUCTIONS** and ensure you fully understand them before using this product.

**Product modification is not permitted.**

**Before use, always check that the sling is not damaged or worn.**

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at [www.humancaregroup.com](http://www.humancaregroup.com).

Contact your local distributor if you have any questions about the product and its use.

## Notice to user/patient in case of serious incident

Any serious incident that has occurred in relation to the product, should be reported to the local contact, who reports to the manufacturer, and the competent authority of the country in which the user/patient is established.

# Rollator Fellow Supreme

## Compliances and Standards

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.  
The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.  
The CE mark is on the product.  
This product has been tested in accordance with ISO 11199-2:2021.

## About our products

### On delivery

Check the following before using your new product:

1. Check that no damage has occurred during transport (if this is the case, please immediately inform the delivery agent).
2. Check that the number and type (s) of products detailed on the delivery certificate agree with the goods delivered.
3. Check that the delivery is of the desired model and type of product.

Please contact us immediately in case of any delivery problem.

### Return or refund

If any product has to be returned for any reason, please contact us to obtain a claim number and advice on how to proceed.

The goods should be returned in complete and undamaged condition in their original packaging.

The following information should accompany any product on its return. The product's identification number, delivery date, the date the user took possession of the aid, the date of the product's last service, the person whom we should contact about the fault, whether it be the user or another, and the nature of the fault.

Human Care HC AB reserves the right to inspect the returned product. If no fault can be found, the item will be returned to the user. Transport costs may then be chargeable.


### Product lifetime

The product's expected lifetime is seven (7) years, if the product is used as intended and maintained according to the manufacturer's instructions, depending on the intensity of use and maximum load applied during use. If the product label is no longer legible, the product should be discarded.

# Product label

The product label contains information about the walker.

The label sits high up on the right front frame.

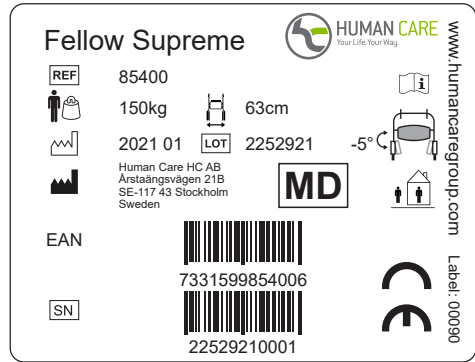ENGLISH

	Article number
	Maximum user weight
	Date of manufacture
	Address of manufacture
	UPC code
	CODE 39 barcode, Serial number

	Read the manual
	Product width
	Indoor / outdoor use
	CE marking
	MD marking

## Additional Warnings

Ensure that all bolts are securely fastened before using the rollator!

DO NOT adjust brake tension until wheels lose molding sheen!

Brakes must be in the parked/locked position before sitting on the rollator!

DO NOT hang anything on backstrap or it may break!

# Rollator Fellow Supreme

## Safety

In the instruction manual, three ways of drawing special attention are used. These ways are highlighted in the following manner.

### Attention!

Risk of personal injury or mechanical fault.

### Important!

Risk of mechanical damage or fault.

### Note!

Important information to ease handling.

## Recycling of products

When recycling, the product's chassis is separated from other parts. At separation, the parts can be sorted according to material after unscrewing and dismantling rivets and other connections. The different parts should be separated into metal, plastic or combined materials.

The products consist mainly of steel pipe and other metal components.

Examples of parts which consist of combined materials, are the plastic front forks (built up in several layers), wheels and other components, which consist of a composite of materials.

### Note!

Different authorities/recycling companies handle recycled materials in different ways. It is thus fitting when in any doubt over the best way to proceed when recycling, to contact the relevant governing body.

## Environment friendliness

Human Care products are manufactured so as to have as little effect on the environment as possible during their lifecycles. This allows for the expired product to be dismantled and for its parts to be recycled. Products with lengthy lifecycles safeguard our sensitive environment and are thus more environmentally friendly.

Human Care HC AB is a part of REPA. REPA – A countywide system for recycling of packaging.

## Intended use

Fellow Supreme is an easily used walking aid which is suitable for use both in- and outdoors. Unfolding and folding are achieved with simply used handles.

## Technical data

Rollator Article no.*	Min.	Max.	Seat height	Width	Seat width	Length	Wheel size	Weight (kg)	L x B x H (Min.)	Standard (kg)
Fellow Supreme										
85400	730	1000	585	630	440	680	195x45	7,9	840x630x370	150

(All measurements in mm)

\* When ordering, make an addition after the part number for the package size, QX, where Q is for quantity and X for the size of the package. Please see [www.humancare.com](http://www.humancare.com) for more information.

# Rollator Fellow Supreme

## Accessories

Accessories		Article no.	
Basket with handles		10666	Max load 10kg.
Tray		10667	Max load 2kg.
Bag		08808	Max load 2kg.
Cane holder		10603	
Back support		10668	
News paper strap		10679	
Oxygen holder		10671	Max load 5kg.
Slow down break		10625	
One-hand break	Left	10669	
	Right	10676	

See respective assembly instruction.

Table 1. Possible accessory combinations

	Basket	Tray	Bag	Cane holder	Back support	Paper rubber band	Oxygen holder	Slow down break	One-hand break
Basket	✓	✗	✓	✗	✗	✗	✓	✗	✗
Tray	✗	✓	✗	✗	✗	✗	✗	✗	✗
Bag	✓	✗	✓	✗	✗	✗	✓	✗	✗
Cane holder	✗	✗	✗	✓	✗	✗	✗	✗	✗
Back support	✗	✗	✗	✗	✓	✗	✗	✗	✗
Paper rubber band	✗	✗	✗	✗	✗	✓	✗	✗	✗
Oxygen holder	✓	✗	✓	✗	✗	✗	✓	✗	✗
Slow down break	✗	✗	✗	✗	✗	✗	✗	✓	✗
One-hand break	✗	✗	✗	✗	✗	✗	✗	✗	✗

✓ = Not applicable.


## User guide

- Read all warnings and instructions carefully before using the rollator.
- Ensure that the rollator is properly unfolded before use.
- The rollator may only be used as a support for walking or sitting.
- Never stand on the rollator.
- Do not use the product on stairs or escalators.
- Be extra careful when using the rollator on slopes or uneven surfaces.
- Never sit on the rollator and roll yourself forward with your feet.
- Distribute weight evenly to prevent the rollator from tipping when you sit on it.
- Always set the parking brake before using the rollator for support while sitting down or getting up.
- Do not place anything on a folded up rollator when it is in storage or being transported as this may cause damage to it.
- Do not store the rollator close to a heat source as this could cause damage to some details.
- Be careful when you handle the rollator if it has been exposed to abnormally high or low temperatures to avoid injuries.
- Do not store the rollator outdoors.
- Save the user instructions for future use.


### Usage

To get the best possible walking space, the seat can be tipped up. This also makes it easier to reach workspaces etc. To increase the walking space even more, the plastic grating can be removed.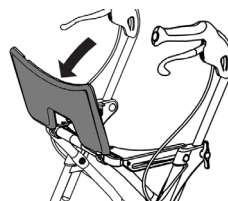

### Tilt help


To make it easier to pass for example a threshold, there are two tilt helps, step on one of them to tilt the rollator.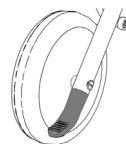

### Unfolding

1. Push the red button.
2. Push on the rollator. The rollator locks automatically.

Important! There are two positions when folding; transport and storage position. To further fold/unfold the rollator, push the red button. Make sure that the wire is not in the way when unfolding.

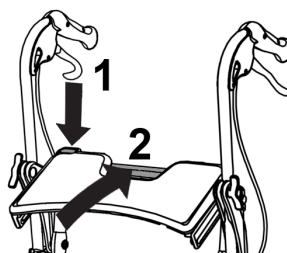
# Rollator Fellow Supreme

## Folding

1. Place your left hand on the left handle.
2. Press the red button with your right thumb while lifting the rollator upwards. Hold on with your left hand.
3. Release the button at standing or folded position and the rollator will lock automatically.

**Warning!** The rollator must not under any circumstances be used as a seat or walk aid when folded.

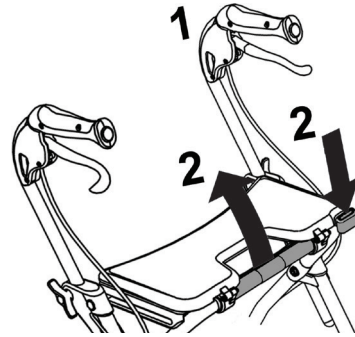
## Handle height adjustment

1. The handle height is adjusted by untightening the wing nut. Push or pull the handle pipe.
2. At requested position tighten the wing nut firmly. Angle the handles 3° inwards toward the centre (equal to 1cm measured from the front edge of the push handles).


**Important!** The handles must not be heightened above the "MAX" mark on the sides of the handle pipes. The handles must be pointing straight forward and the handle pipe numbers must be aligned with indication to avoid unstableness and dangerous situations.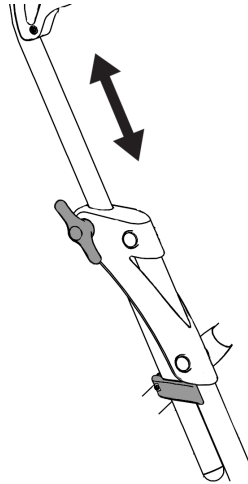

**Note!** The rollator has two handle height memories that shall be pulled upwards and tightened when the handles are in correct positions. They help you to easily regain the correct handle position after for example transportation. The handle height memories works up to a height of 96 cm. When higher handle heights are used, remove the handle height memories. Push down the handles to reduce the rollator size during transportation. The angle between the lengthwise line of the handles and the forward direction of the rollator must always be 3°.

**Warning!**  
Clamp risk!

**Warning!**  
Clamp risk!

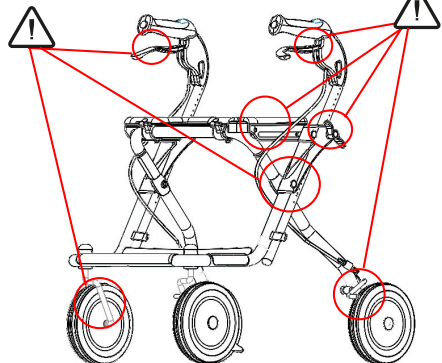

## Parking brake

Push brake lever down to brake when stationary.


**Important!** Engage parking brake before sitting.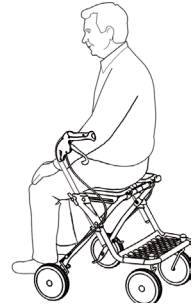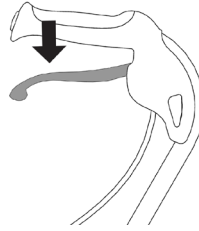

## Brake

Pull brake lever up to brake whilst travelling.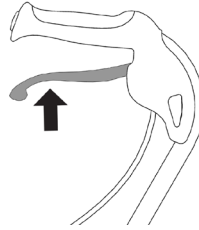


**Note!** The brakes consists of one brake handle for each wheel, where the right brake handle affects the right wheel, and the left brake handle affects the left wheel. The brake handle pulls a wire that runs in a wire housing. The wire pulls a brake gear that affects the wheel. When a single hand brake is used, one brake handle affects both wheels.

## Brake adjustment

Loosen locking nut; adjust brake by loosening/tightening the adjustment casing. After adjustment tighten locking nut.

**Note!** Continuous braking leads to excessive wear of wheels and brakes.

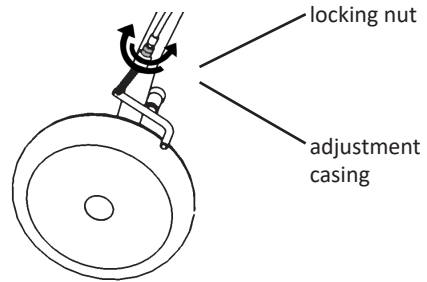# Rollator Fellow Supreme

## Cleaning and service

### Important!

When changing parts of the rollator, use only Human Care parts to ensure continued CE validation.

### Important!

Do not alter the rollator yourself.

### Important!

Check the rollator regularly to ensure its continued safety to the user.

Regular checks should be carried out as recommended by the instructions detailed in art. No. 99209.

For more information on servicing, reconditioning, spare parts and accessories, see Human Care's website: [www.humancaregroup.com](http://www.humancaregroup.com) or contact us at the address, phonenumber or e-mail address according to back page.

## Warranty Terms

---

The Human Care HC AB warranty covers defects in material and manufacture of the walking aid.

Wear on wheels, brake parts and handles are not included. Finish damages which have occurred after delivery are not covered. The warranty presumes that the product has been used in the intended way and that it has been regularly reconditioned in accordance with our reconditioning instructions. The ID number of the product must also be identifiable. Furthermore, the product must not have been modified or customized by any party other than Human Care HC AB. The warranty period is one (1) year or according to agreement.

For complaints, please contact us for instructions on how to proceed.

For complaints, the following information should be indicated: product ID number; the date of the last reconditioning; the name of the consignor contact; a description of the fault.

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