# HUMAN CARE

Your Life. Your Way.

## Rollator Fellow Classic

89009, 89109, 89309, 89509, 89609

Bruksanvisning/User Manual/Brukermanual/Brugermanual/  
Ohjekirja/Gebruikshandleiding

## Innehåll

### SVENSKA

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## User Manual

### Important!

You must read the user manual for your device prior to usage. Keep this booklet and information handy for future use.

## Symbols

**WARNING** symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

**INFORMATION** symbol indicates recommendations and information for proper, trouble-free usage.

**WEIGHT CAPACITY** symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

**CE MARKING** symbol indicates product meets requirements of EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**MEDICAL DEVICE** symbol indicates that the product is a medical device according to the definition of medical device in EU Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR).

**RECYCLE** according to local regulations.

**READ INSTRUCTIONS** and ensure you fully understand them before using this product.

***Product modification is not permitted.***  
***Before use, always check that the sling is not damaged or worn.***

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at [www.humancaregroup.com](http://www.humancaregroup.com).

Contact your local distributor if you have any questions about the product and its use.

### Notice to user/patient in case of serious incident

Any serious incident that has occurred in relation to the product, should be reported to the local contact, who reports to the manufacturer, and the competent authority of the country in which the user/patient is established.

## Compliances and Standards

Human Care is an ISO 13485:2016 certified Swedish medical device company. The Quality Management System is in compliance with US 21 CFR part 820.

The product is CE marked in accordance with EU

Directive 93/42/EEC (MDD) and/or EU Regulation 2017/745 (MDR), as class I medical device.

The CE mark is on the product.

This product has been tested in accordance with ISO 11199-2:2021.

## About our products

### On delivery

Check the following before using your new product:

1. Check that no damage has occurred during transport (if this is the case, please immediately inform the delivery agent).
2. Check that the number and type (s) of products detailed on the delivery certificate agree with the goods delivered.
3. Check that the delivery is of the desired model and type of product.

Please contact us immediately in case of any delivery problem.

### Return or refund

If any product has to be returned for any reason, please contact us to obtain a claim number and advice on how to proceed.

The goods should be returned in complete and undamaged condition in their original packaging.

The following information should accompany any product on its return. The product's identification number, delivery date, the date the user took possession of the aid, the date of the product's last service, the person whom we should contact about the fault, whether it be the user or another, and the nature of the fault.

Human Care HC AB reserves the right to inspect the returned product. If no fault can be found, the item will be returned to the user. Transport costs may then be chargeable.

### Product lifetime

The product's expected lifetime is seven (7) years, if the product is used as intended and maintained according to the manufacturer's instructions, depending on the intensity of use and maximum load applied during use. If the product label is no longer legible, the product should be discarded.

# Rollator Fellow Classic

## Product label

The product label contains information about the rollator.

The label is located on the backside of the seat.

	Article number
	Maximum user weight
	Date of manufacture
	Address of manufacture
EAN	UPC code
	CODE 39 barcode, Serial number

	Read the manual
	Product width
	Indoor / outdoor use
	CE marking
	MD marking

## Additional Warnings

Ensure that all bolts are securely fastened before using the rollator!

DO NOT adjust brake tension until wheels lose molding sheen!

Brakes must be in the parked/locked position before sitting on the rollator!

DO NOT hang anything on backstrap or it may break!

## Safety

In the instruction manual, three ways of drawing special attention are used. These ways are highlighted in the following manner.

### Warning!

Risk of personal injury or mechanical fault.

### Important!

Risk of mechanical damage or fault.

### Note!

Important information to improve handling.

## Recycling of products

When recycling, the product's chassis is separated from other parts. At separation, the parts can be sorted according to material after unscrewing and dismantling rivets and other connections. The different parts should be separated into metal, plastic or combined materials.

The products consist mainly of steel pipe and other metal components.

Examples of parts with consist of combined materials, are the front fork including bearings wheel with bearing.

### Note!

Different authorities/recycling companies handle recycled materials in different ways. It is thus fitting when in any doubt over the best way to proceed when recycling, to contact the relevant governing body.

## Environmentally friendly

Human Care HC AB aids are manufactured to have as little effect on the environment as possible during their lifetime. This allows for the used product to be dismantled and for its parts to be recycled. Products with lengthy lifecycles safeguard our sensitive environment and are thus more environmentally friendly.

Human Care HC AB is a part of REPA. REPA – A countrywide system for recycling of packaging.

# Rollator Fellow Classic

## Intended use


Fellow Classic is an easily used walking aid which is suitable for use both in- and outdoors.

Unfolding and folding are achieved with simply used handles.

This safety walker is available in five varieties with differing heights and widths. See below.

## Technical data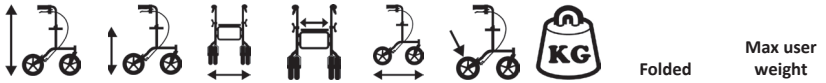

Rollator Article no.*	Min.	Max.	Seat height	Width	Seat width	Length	Wheel size	Weight (kg)	LxBxH (Min.)	Standard (kg)
Fellow Classic										
89009 Standard	770	950	590	630	440	700	200x40	8,0	860x630x290	150
89109 Low	680	805	505	620	440	610	200x40	7,7	760x620x290	140
89309 Wide	770	950	600	690	500	700	200x40	8,7	850x690x290	150
89509 Narrow	770	950	600	595	440	700	200x40	8,1	850x595x290	130
89609 High	820	1000	640	625	440	700	200x40	8,1	910x625x290	130

(All measurements in mm)

\* When ordering, make an addition after the part number for the package size, QX, where Q is for quality and X for the size of the package. Please see [www.humancaregroup.com](http://www.humancaregroup.com) for more information.

## Accessories

Accessories	Model	Article no.	
Basket with handles	Standard	08805	Max load 10 kg
	Low	88051	
	Wide, Narrow, High	88052	
Tray		08806	Max load 2 kg
Bag	Standard, Narrow, Wide	08808	Max load 2 kg
	Low	88081	
Cane holder		10603	
Back support		10668	
Oxygen holder		08811	Max load 5 kg
Slow down break		10625	
IV stand	Standard	08814	Max load 3 kg
One-hand break	Right	10677	
	Left	10678	
Cerb climber		08813	

See respective assembly instruction.

Table 1. Possible accessory combinations

	Basket	Tray	Bag	Cane holder	Back support	Oxygen holder	Slow down break	IV stand	One-hand break	Cerb climber
Basket	✓	✗	✓	✗	✗	✓	✗	✓	✗	✗
Tray	✗	✓	✗	✗	✗	✗	✗	✓	✗	✗
Bag	✓	✗	✓	✗	✗	✓	✗	✓	✗	✗
Cane holder	✗	✗	✗	✓	✗	✗	✗	✓	✗	✗
Back support	✗	✗	✗	✗	✓	✗	✗	✓	✗	✗
Oxygen holder	✓	✗	✓	✗	✗	✓	✗	✓	✗	✗
Slow down break	✗	✗	✗	✗	✗	✗	✓	✗	✗	✗
IV stand	✓	✓	✓	✓	✓	✓	✗	✓	✗	✓
One-hand break	✗	✗	✗	✗	✗	✗	✗	✗	✗	✗
Cerb climber	✗	✗	✗	✗	✗	✗	✗	✓	✓	✓

✓ = Not applicable.


# Rollator Fellow Classic

## User guide


- Read all warnings and instructions carefully before using the rollator.
- Ensure that the rollator is properly unfolded before use.
- The rollator may only be used as a support for walking or sitting.
- Never stand on the rollator.
- Do not use the product on stairs or escalators.
- Be extra careful when using the rollator on slopes or uneven surfaces.
- Never sit on the rollator and roll yourself forward with your feet.
- Distribute weight evenly to prevent the rollator from tipping when you sit on it.
- Always set the parking brake before using the rollator for support while sitting down or getting up.
- Do not place anything on a folded up rollator when it is in storage or being transported as this may cause damage to it.
- Do not store the rollator close to a heat source as this could cause damage to some details.
- Be careful when you handle the rollator if it has been exposed to abnormally high or low temperatures to avoid injuries.
- Do not store the rollator outdoors.
- Save the user instructions for future use.

### Unfolding

Pull the locking frame under the seat until the rollator automatically unfolds and locks.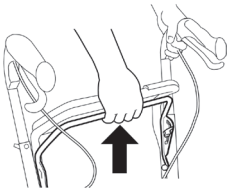


**Important!** Check that the red catch falls of its own weight.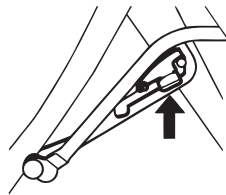

### Folding

1. Lift red catch.
2. Pull locking frame up. The rollator folds automatically.
3. Fold stand together until the rollator locks folded.

**Warning!**  
Clamp risk!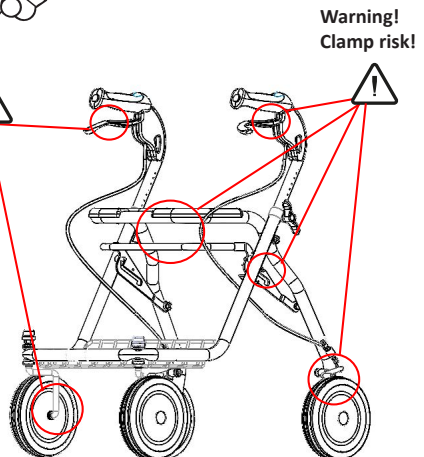

**Warning!**  
Clamp risk!

### Adjustment of handles


Loosen wing nuts and adjust the handles to wrist height. Tighten the wing nuts.

Sideways angle of the handles : 3° inwards.

### Important!

handles must not be heightened above the "MAX" mark on the sides of the handle pipes. The handles must be pointing straight forward. The handle pipe numbers must be aligned with the slit at the side of the pipes to avoid unstableness and dangerous situations.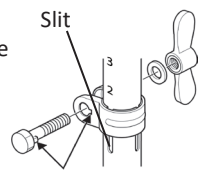

### Warning!

The handles must not be pointing outwards because of increased tip risk.

On assembly of the wingnut component, it is important that the base of the screw head engages fully the groove in the recess.

### Parking brake

Push brake lever down to brake when stationary.



### Important!

Engage parking brake before sitting.

### Brake

Pull brake lever up to brake whilst travelling.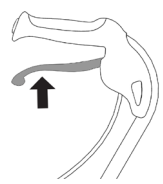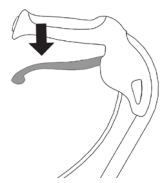

### Note!


The brakes consists of one brake handle for each wheel, where the right brake handle affects the right wheel, and the left brake handle affects the left wheel. The brake handle pulls a wire that runs in a wire housing. The wire pulls a brake gear that affects the wheel. When a single hand brake is used, one brake handle affects both wheels.

### Brake adjustment

Loosen locking nut; adjust brake by loosening/ tightening the adjustment casing. After adjustment tighten locking nut.

**Note!** Continuous braking leads to excessive wear of wheels and brakes.

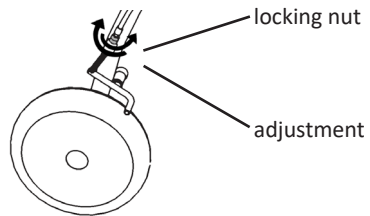# Rollator Fellow Classic

## Cleaning and service

### **Important!**

When changing parts of the rollator, use only Human Care parts to ensure continued CE validation.

### **Important!**

Do not alter the rollator yourself.

### **Important!**

Check the rollator regularly to ensure its continued safety to the user.

Regular checks should be carried out as recommended by the instructions detailed in 99210.

For more information on servicing, reconditioning , spare parts and accessories, see Human Care's website: [www.humancaregroup.com](http://www.humancaregroup.com) or contact us at the address, phonenumber or e-mail address according to back page.

## Warranty

---

Human Care HC AB has high demands on our product quality and our warranty cover material and manufacturing faults of our products. The warranty is not covering wear and tear on wheel, brakes, handles grips and surface damages arisen after delivery. The warranty is valid if the product is used in the intended way and the maintenance instructions have been followed. The product identity needs to be readable. The product shall not have been modified or changed in any way after the delivery from Human Care. The warranty time is 1 year if noting else has been stated in contract.

In case of product complaint please contact us for further instructions.

The following information is needed to create a product complaint:

Id number of the product, date for when the last maintenance were performed, customer contact person and failure description.

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The product label for the Fellow Classic rollator contains the following information:

- Fellow Classic** (Product Name)
- HUMAN CARE** logo with tagline "Your Life. Your Way."
- REF** 88009 (Reference number)
- 150kg** (Maximum user weight)
- 63cm** (Product width)
- 2021 01** (Date of manufacture)
- Lot** 2252921 (Lot number)
- 0°C** (Temperature rating)
- MD** (MD certification mark)
- Human Care HC AB**, Arstaängsvägen 21B, SE-117 43 Stockholm, Sweden (Manufacturer address)
- EAN** 7331599880098 (UPC code)
- SN** 22529210001 (Serial number)
- Website: [www.humancaregroup.com](http://www.humancaregroup.com)
- Label number: 00090

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