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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. The strain on the user and the patient / care-dependent can be reduced significantly through the use of transfer and raising aids.

The use of the Raising Aid Pegasus should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Raising Aid Pegasus is used to assist a patient with getting up after a fall or when raising from a low level. This product is used in particular in home care and other health care facilities as well as hospitals.

The product may only be used for the specified purpose and under the specified conditions.

The information in this user manual is for the patient who is being transferred and moved, as well as the user assisting the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: PM-4222 Raising Aid Pegasus

1.2 Accessories

Item-no.: EZM-4628 handle

1.3 Scope of delivery

Raising Aid Pegasus – basic device
removable seat back with handle holder
handle
user manual

1.4 Storage, transport

Store/transport at -5°C – 40°C
Other: protect from sunlight

1.5 Service life / lifecycle

If the product is used in accordance with the safety instructions, then the expected product service life is **ten years**. However, the effective service life may vary depending on the frequency and intensity of use and the number of cleaning and disinfection cycles.

Perform regular function checks on the product and be sure to observe the safety information (see item 2).

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer/lift and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Weight limits must be observed (135 kg).
- Application without assistance may only be carried out by persons with sufficient physical capability. Prior training with authorised qualified personnel is required.
- Ensure that the seat and backrest can move freely during use
- Before raising the patient with the Raising Aid Pegasus, ensure that the patient is not suffering from any injuries or weakness which would require calling a doctor or ambulance and at the same time precludes use of the device.
- The Raising Aid Pegasus may only be used by authorised persons, or respectively by persons who have been trained in use of the device and who are familiar with the patient's medical condition.

2.1 Symbols used

	Wipe off		max. load capacity see item 6
	Do not bleach		Manufacturer
	Do not tumble		Batch no. / lot no.
	Do not iron		Identification code
	Do not dry clean		Conformity mark
	Warning		Item number
	Observe the user manual		Medical device label

2.2 Function check

The Petermann Raising Aid Pegasus is designed with a stainless steel frame and therefore doesn't require any special maintenance. A visual inspection for defects is required before each use. The device may not be used if there are any noticeable defects.

Periodic inspections should be performed at least once per year; i.e. visual inspection of the load-bearing parts (function test). For sanitary reasons, the device must be cleaned regularly.

➤ **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since Raising Aid Pegasus is suitable for all applications involving people who require assistance.

This product may be reused.

The frequency of further use depends on the respective condition of the device. On the basis of the MPG (German Medical Devices Act) "Processing of Medical Devices" § 4, para. 2 MPBetreibV (Ordinance on Operators of Medical Devices).

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- Insufficient trunk stability
- Acute injuries that cannot be assessed



3.3 Application risks

- Safety contact between the patient and user must be maintained during the transfer process → Risk of falling.
- Incorrect operation (application) → risk of skin injury
- Incorrect operation (handle) → risk of crushing
- Pegasus stops in the room → risk of stumble

3.4 Areas of application

The product can be used in home care and other health care facilities.

3.5 Application illustration

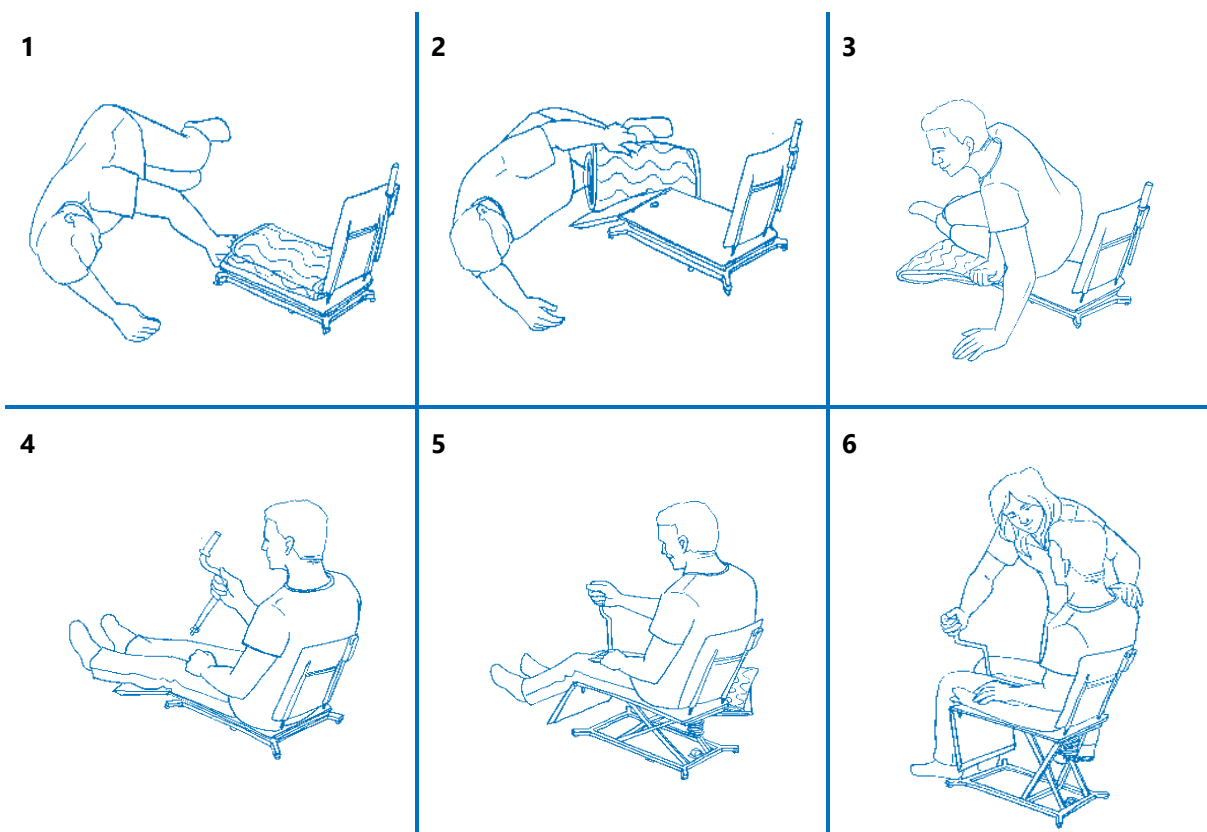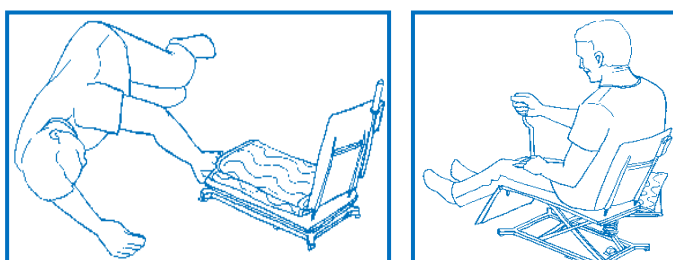3.6 Application

3.6.1 General description

The Raising Aid Pegasus aids mobilisation from the floor to a seated position. Depending on the clinical picture, this can be performed alone or with assistance. The seat of the Raising Aid Pegasus has a minimum receiving height of 80 mm.

The Raising Aid Pegasus may be used exclusively as a Raising Aid in accordance with the intended use. Any use other than the intended use is prohibited. Training in use of the Raising Aid Pegasus may only be provided by authorised, qualified personnel.

3.6.2 Setup

- Set the base up.
- Place the included hand crank in the crank receiver on the backrest.
- The Raising Aid Pegasus can be raised on all sides of the seat.
- Insert the backrest into the backrest receiver.
- Optional use without the backrest is possible.

3.6.3 Application with assistance

- Set the Pegasus up within reach.
- Fold the sliding plate down from the seat plate and onto the floor. →to create an incline
- Ready the Raising Aid Quick, slide and anti-slip mat. (included in the set PM-4225)
- Turn the patient onto their side.
- Place the slide under the buttocks.
- Turn the patient in a supine position.
- Raise the legs.
- Place the anti-slip mat under the patient's feet.
- Place the Raising Aid Quick against the buttocks.
- Place the handles of the Raising Aid Quick across one another on the patient's stomach.
- Move the patient into a sitting position over the shoulder with a slight turning motion.
- Secure the patient by the shoulder.
- With the other hand, reach into the opening on the inclined surface of the Raising Aid Pegasus.
- Pull the Raising Aid Pegasus under the patient's buttocks by the inclined surface.
- Take hold of the Raising Aid Quick handles.
- If necessary, support the user's knee on the seat of the Raising Aid Pegasus. (Observe max. weight load!)
- Pull the patient onto the seat by shifting you weight back.
- Remove the slide from under the patient.
- Insert the crank into the hex receiver.
- Turn to the right →Raising Aid Pegasus is raised.
- Maximum number of turns: 57 (from the highest to the lowest position)
- Move to the desired height.
- Transfer to a wheelchair is performed in accordance with the clinical picture.

3.6.4 Application without assistance

With sufficient physical capability, the Pegasus can also be used without assistance. Prior training with authorised, qualified personnel is required in this case.

3.6.5 Place the Raising Aid Pegasus in the starting position

- Fold the inclined surface onto the seat.
- Turn to the left →The Raising Aid Pegasus is lowered.
- **Due to the counterbalance springs, lowering is easier if weight is applied.**

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.6.6 Behaviour in case of a fault

The following information helps you with troubleshooting:

Fault	Possible cause	Fault resolution
Raising Aid Pegasus moves with difficulty or not at all	Foreign body is blocking the mechanism	Remove foreign body
Spindle or spindle nut doesn't move freely	Foreign body is blocking the mechanism	Remove foreign body
Doesn't stand securely without slipping	Rubber feet severely soiled or damaged	Clean rubber feet or replace if necessary
Sprocket doesn't turn	Safety pin is broken	Repair by qualified personnel is required

If you are unable to rectify the fault despite this guide, please contact your responsible medical supply store or call our service number: **+49 (0) 9868 9339-0**.

3.7 Approved accessories, combination with other products

Raising Aid Pegasus is used in combination with the following products:

- PM-7010/7011 Non-Slip Mats
- PM-6080 Raising Aid Quick
- Blue Motion Slide

Petermann Raising Aid Pegasus is part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

3.7.1 Sets


Item no. set	Ingredients set
PM-4225	PM-4222 Raising Aid Pegasus
	PM-6080 Raising Aid Quick
	BM-0652 Blue Motion Slide

Please observe the user manuals of the other items as well.

4 Product marking / CE

The Petermann Raising Aid Pegasus is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

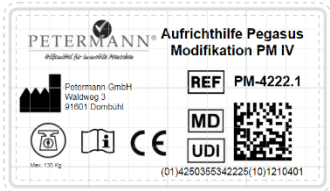5 Cleaning, care and disinfection

The Petermann Raising Aid Pegasus is made from rust-proof components and therefore doesn't require any special maintenance. A visual inspection for defects is required before each use. The device may not be used if there are any noticeable defects. Therefore, please contact your responsible medical supply store or call our service number. In accordance with the hoist standard (DIN EN ISO 10535: 2006), a periodic inspection must be performed at least once a year; i.e. visual inspection of the load-bearing parts and the lifting mechanism (function test). Please contact your medical supply store in this regard.

For sanitary reasons, please clean the device on a regular basis. You can use common liquid household cleaner. After cleaning, all moving parts such as the threaded rod, gears, etc. can be coated with an oil-free lubricant (Vaseline) using a brush.

If you do not wish to perform the inspection, cleaning and disinfection of your Raising Aid Pegasus yourself, you can send your device to our shop.

The device can be cleaned and/or disinfected as follows before being used again:

Cleaning / disinfection of the device as follows is recommended:

- Place the Raising Aid Pegasus without the crank and accessories in a suitable tub with lid. Fill with suitable cleaning solution (Refer to the detergent specifications for mixing ratio and residence time).
- Final cleaning with pressure washer. Addition of a disinfectant leads to simultaneous disinfection of the device.
- Rinsing and drying
- Visual inspection for cleanliness and integrity (e.g. material properties, corrosion)
- Function test of the frame / scissor and spring mechanism
- Acoustic check: unusual noises during raising or lowering?
- Inspect the seat and backrest
- Repeat the cleaning and disinfection process, if necessary
- Test run and verification of functionality under maximum load
- Inspection of the accessories as well as suction cup, crank for proper functionality
- General repair, replacing of accessories, if necessary
- Final function test of the device
- In the event of wear or damage to the device which cannot be repaired, the Raising Aid Pegasus must be taken out of service!

Please let us know of any damage or defects on the device which are not attributable to normal wear and tear, so that we can implement corrective measures if necessary.

6 Technical Data

Item no.	Material	Colour	Load-bearing capacity
PM-4222	Drive mechanism: <ul style="list-style-type: none"> ➤ - high grade stainless steel and spring steel Seat: <ul style="list-style-type: none"> ➤ Polypropylene Back: <ul style="list-style-type: none"> ➤ Polypropylene Handle: <ul style="list-style-type: none"> ➤ high grade stainless steel 1.4301 Handle box: <ul style="list-style-type: none"> ➤ plastic white Frame: <ul style="list-style-type: none"> ➤ high grade stainless steel 1.4301 	white	135 kg

Item no.	Description	Dimensions in mm	Weight	UDI
PM-4222	Raising Aid Pegasus	Standing 360 x 640 included back 500	9,4 kg basic device	(01)04250355352225(10)x
	Seat height above:	420 on load		
	Seat height below:	80 on load		
	Lift height	330		
	Seat size	350 x 560 + 350 x 340 Fold-out		
	Back	420 x 320	1 kg	

The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The warranty period is 3 years and covers only the height adjustable Raising Aid Pegasus.

Please note that the warranty does **not** apply to:

- Repairs which were not performed by Petermann or authorised Petermann dealers.
- Damage to or failure of the Raising Aid Pegasus which were caused by improper use.
- Accidental or indirect injury and/or damage to property.

8 Disposal information

There are no specific disposal regulations for Raising Aids Pegasus, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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