
Kenmak[®]

part of the Human Care Group

producing technology for health

Instruction for Usage of Kenmak Hospital Furniture Model: K058 PTS

TILTING PATIENT DINING TABLE

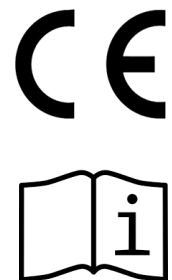Read user manual before using the product!

Table of Contents

- 1- Introduction**
- 2- Intended Use**
- 3- Warnings**
- 4- General Features and Accessories**
- 5- Assembly Instructions**
- 6- Technical Details**
- 7- Table Safety Devices**
- 8- Instructions for Use**
- 9- Spare Parts**
- 10- Product Labelling**
- 11- Environment, Transport and Storage**
- 12- Maintenance and reconditioning**
- 13- Warranty / Life age**

1- Introduction

Thank you for choosing your hospital furniture from Kenmak. Every single Kenmak Hospital Furniture product is being produced under high quality standards bearing in mind how to minimize the risk of environmental reverse effect. Kenmak Hospital Furniture's operation are certified by globally recognized authorities according to ISO 9001, ISO 13485 and ISO 14001 standards.

Kenmak Hospital Furniture products are manufactured in Turkey in line with aforementioned quality standards and are tested before dispatching the factory by our Quality Team.

Any incident involving the product should be reported to the manufacturer and a relevant regulatory body.

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www.kenmak.com.tr

2- Intended Use

K058-PTS Kenmak tilting patient dining table is produced with intend to be used in hospitals, nursing homes, and homes.

Product should be used in suitable indoor atmosphere, between 0 to +40 degrees.

It is intended to be used by individuals older than 12 years old.

The product should only be used with accessories delivered together with the table or with accessories produced by Kenmak.

For the information on maximum loading capacity see the “Technical Details” section of this instruction manual.

3- Warnings

Using the equipment for purposes other than those previously described is forbidden. The equipment must only be used by properly trained person.

Before starting to use the table, the operator must be perfectly aware of how the table works and of all technical-functional features. Such awareness can be acquired by carefully reading this manual.

If it is possible, the nursing staff should properly inform patients on how the table works and should explain the controls so that the patient can operate the product. After such information has been provided, if it is possible, the patient should be asked to "trial run" the table.

Removing any safety or protection devices on the appliance is strictly forbidden; periodically, at weekly intervals, check the integrity of such devices and make sure they are working properly.

Before carrying out any maintenance or repair jobs on the equipment, the operator should have carefully read and understood the technical information provided in this manual. All maintenance and repair jobs must be done by skilled personnel.

Never position the equipment close to heat sources.

Position and secure the equipment in a stable manner on a horizontal floor. The table movement area must be kept clean, in order and devoid of objects that could restrict free movement. The instructions, accident-prevention rules and warnings contained in this manual must always be followed.

4- General Features and Accessories

- Height adjustment by the help of a hangrip
- Tilttable eating/reading board
- 50 mm diameter double castors
- Brakes on 2 castors
- Antibacterial ABS plastic material eating/reading board
- Electrostatic coated metal frame

5- Assembly Instructions

The product is delivered fully assembled.

6- Technical Details

- Possible table positions:

Height and table tilt functions are adjustable by handgrips.

- Height Range: 680-1000 mm
- Eating Board Tilting Angle: (-40)° - (+40)°
- Eating Board Length: 820 mm
- Eating Board Width: 380 mm
- Castor Diameter: 50 mm
- Maximum Loading Capacity: 20 kg

7- Table Safety Devices

Handgrip for Height Adjustment

When loosened, the eating/reading board can be moved up and down.

Castor Brakes

Prevents the table from accidentally moving while the patient is eating or reading.

8- Instructions for Use

Before using the table, carefully read all the instructions in this user manual. Correct control learning will ensure proper and safe table use.

Prior to enabling the various table movements, make sure that the patient bed and table are in convenient positions.

Locking the wheels

The bed is equipped with 2 diagonally lockable wheels. Press on the wheel pedals to lock.

Pressed pedal = locked

Released pedal = unlocked

9- Spare Parts

Eating/Reading Board

Handgrip for Height Adjustment

Castors

Please visit www.kenmak.com.tr for additional accessories.

10- Product Labelling

Kenmak®
part of the Human Care Group

PRODUCER:
KENMAK HOSPITAL FURNITURE & EQUIPMENTS Tel:+90 232 274 17 17 (pbx)
Fax:+90 232 274 67 77
www.kenmak.com.tr
kenmak@kenmak.com.tr
Sarıç Yolu No:23 Gaziemir-İZMİR-TURKEY / (Made in Turkey)
Model Name : K058-PTS
Medical Device Class: I

Kenmak Hospital Furnitures & Equipments
Sarıç Yolu No:23 Gaziemir-İZMİR-TURKEY
Production Date : 05.08.2021

Serial Number :
Weight : 10kg
Dimensions : 82x38x68-100(H)cm
- Authorized Representative in Europe:
Human Care HB AB
Årstaängsvägen 21B 117 43, Stockholm / SWEDEN
T: +46 8 510 132 00; F: +46 8 665 35 10
info.se@humancaregroup.com

TS E **CE** **MD**

11- Environment, Transport and Storage

Environment

Kenmak is certified in accordance with ISO 14001 and work with continuous improvement to reduce total environmental impact. A primary focus is logistics where we have reduced our total environmental footprint.

Our products are recyclable. Help our environment and hand in the product for recycling. Follow local instructions for recycling.

Transport and Storage

Conditions for transport and storage:

- Air humidity between 10 and 80% RH
- Temperature between -300 and 400C
- Pressure between 700 and 1100 hPa

12- Maintenance and reconditioning

The table is designed to require the minimum maintenance. It is recommended that preventive maintenance be carried out according to our service schedule every other year.

Normal Cleaning

Mechanics cleaned with a damp cloth. Readily available cleansers and disinfectants can be used. Can be rinsed with water.
The table base and eating/reading board are cleaned with a damp cloth. Readily available cleansers and disinfectants can be used.

High-pressure Wash

Before washing castors have to be in locked position.

13- Warranty / Lifeage

Warranty Time: 2 Years

Expected Lifeage: 10 Years (In case used properly with maintenance program)

Date: 23.08.2021

User Manual Number: K058-PTS / R.210823

Please contact us for any questions:

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