# Kenmak<sup>®</sup>

part of the Human Care Group

*producing technology for health*

**Instruction for Usage of**  
**Kenmak Hospital Furniture**  
**Model: K058 PTS &**  
**K058 PTS-C**

**TILTING PATIENT DINING TABLE**

**CE**

Read user manual before using the product!

## Table of Contents

- 1- Introduction**
- 2- Intended Use**
- 3- Warnings**
- 4- General Features and Accessories**
- 5- Assembly Instructions**
- 6- Technical Details**
- 7- Table Safety Devices**
- 8- Instructions for Use**
- 9- Spare Parts**
- 10- Product Labelling**
- 11- Environment, Transport and Storage**
- 12- Maintenance and reconditioning**
- 13- Warranty / Life age**

# 1- Introduction

Thank you for choosing your hospital furniture from Kenmak. Every single Kenmak Hospital Furniture product is being produced under high quality standards bearing in mind how to minimize the risk of environmental reverse effect. Kenmak Hospital Furniture's operation are certified by globally recognized authorities according to ISO 9001, ISO 13485 and ISO 14001 standards.

Kenmak Hospital Furniture products are manufactured in Turkey in line with aforementioned quality standards and are tested before dispatching the factory by our Quality Team.

Any incident involving the product should be reported to the manufacturer and a relevant regulatory body.

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[www.kenmak.com.tr](http://www.kenmak.com.tr)

## 2- Intended Use

K058PTS and K058PTS-C Kenmak tilting patient dining table is produced with intend to be used in hospitals, nursing homes, and homes.

Product should be used in suitable indoor atmosphere, between 0 to +40 degrees.

It is intended to be used by individuals older than 12 years old.

The product should only be used with accessories delivered together with the table or with accessories produced by Kenmak.

For the information on maximum loading capacity see the "Technical Details" section of this instruction manual.

## 3- Warnings

Using the equipment for purposes other than those previously described is forbidden. The equipment must only be used by properly trained person.

Before starting to use the table, the operator must be perfectly aware of how the table works and of all technical-functional features. Such awareness can be acquired by carefully reading this manual.

If it is possible, the nursing staff should properly inform patients on how the table works and should explain the controls so that the patient can operate the product. After such information has been provided, if it is possible, the patient should be asked to "trial run" the table.

Removing any safety or protection devices on the appliance is strictly forbidden; periodically, at weekly intervals, check the integrity of such devices and make sure they are working properly.

Before carrying out any maintenance or repair jobs on the equipment, the operator should have carefully read and understood the technical information provided in this manual. All maintenance and repair jobs must be done by skilled personnel.

Never position the equipment close to heat sources.

Position and secure the equipment in a stable manner on a horizontal floor. The table movement area must be kept clean, in order and devoid of objects that could restrict free movement. The instructions, accident-prevention rules and warnings contained in this manual must always be followed.

## 4- General Features and Accessories

- Height adjustment by the help of a handgrip
- Tilttable eating/reading board
- K058PTS: 50 mm diameter castors
- K058PTS: Brakes on 2 castors
- K058PTS-C: 75 mm diameter double capsulated castors
- K058PTS-C: Brakes on 4 capsulated castors
- Antibacterial ABS plastic material eating/reading board
- Electrostatic coated metal frame

## 5- Assembly Instructions

The product is delivered fully assembled.

## 6- Technical Details

- Possible table positions:

*Height and table tilt functions are adjustable by handgrips.*

- |                               |                 |
|-------------------------------|-----------------|
| - Height Range:               | 680-1000 mm     |
| - Eating Board Tilting Angle: | (-40)° - (+40)° |
| - Eating Board Length:        | 820 mm          |
| - Eating Board Width:         | 380 mm          |
| - K058PTS Castor Diameter:    | 50 mm           |
| - K058PTS-C Castor Diameter:  | 75 mm           |
| - Maximum Loading Capacity:   | 20 kg           |
| - Product weight:             | 11,6 kg         |

## 7- Table Safety Devices

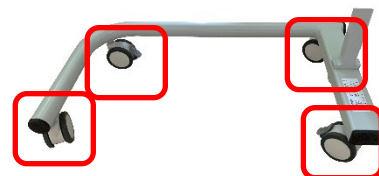
**Handgrip for Height Adjustment**

**Handgrip for Tilting Adjustment**

**Castor Brakes (K058PTS with 2 lockable castors / K058PTS-C with 4 lockable castors)**

## 8- Instructions for Use

Before using the table, carefully read all the instructions in this user manual. Correct control learning will ensure proper and safe table use.

Prior to enabling the various table movements, make sure that the patient bed and table are in convenient positions.

### **Locking the wheels**

The bed table is equipped with 2 diagonally lockable castors. K058PTS-C is equipped with 4 lockable capsulated castors. Press on the wheel pedals to lock.

Pressed pedal = locked

Released pedal = unlocked

## 9- Spare Parts

## Spare part list:

Product name	Article number	Image Ref.	Comments
Eating/Reading board	K058-PT/TT	1	Both models
Handgrip for height adjustment	K058PTS/HIGHTADJ	2	Both models
Capsulated wheel lock (75 mm)	K058-CWL	3	K058PTS-C
Capsulated wheel (75 mm)	K058-CW	4	
Castor no locking (50 mm)	K058PTS/Castornolock	5	K058PTS
Castor with locking (50 mm)	K058PTS/Castorlock	6	K058PTS

Please visit [www.kenmak.com.tr](http://www.kenmak.com.tr) for additional accessories.

## 10- Product Labelling

**Kenmak®**  
part of the Human Care Group

**PRODUCER:**  
KENMAK HOSPITAL FURNITURE & EQUIPMENTS Tel:+90 232 274 17 17 (pbx)  
Fax:+90 232 274 67 77  
www.kenmak.com.tr  
kenmak@kenmak.com.tr  
Sarıç Yolu No:23 Gaziemir-İZMİR-TURKEY / (Made in Turkey)  
Model Name : K058-PTS  
Medical Device Class: I

**Kenmak Hospital Furnitures & Equipments**  
Sarıç Yolu No:23 Gaziemir-İZMİR-TURKEY  
Production Date : 05.08.2021

**Serial Number :**  
Weight : 10kg  
Dimensions : 82x38x68-100(H)cm  
- Authorized Representative in Europe:  
Human Care HB AB  
Årstaängsvägen 21B 117 43, Stockholm / SWEDEN  
T: +46 8 510 132 00; F: +46 8 665 35 10  
info.se@humancaregroup.com

**TS E** **CE** **MD**

## 11- Environment, Transport and Storage

### Environment

Kenmak is certified in accordance with ISO 14001 and work with continuous improvement to reduce total environmental impact. A primary focus is logistics where we have reduced our total environmental footprint.

- Because the top table, height adjustment handle and tilt adjustment handle are made of plastic, it can be reused in plastic recycling.
- Metal body parts as upper table lower connection piece, middle body inner profile, middle body outer profile and lower tube profile can be sent to metal recycling.
- The wheels can be sent for separation first, and then the metal and rubber parts can be sent to the relevant recycling separately.
- Packaging materials should be sent to paper recycling and nylon recycling.

Our products are recyclable. Help our environment and hand in the product for recycling. Follow local instructions for recycling.

### Transport and Storage

Conditions for transport and storage:

- Air humidity between 10 and 80% RH
- Temperature between -30° and 40°C
- Pressure between 700 and 1100 hPa

## 12- Maintenance and reconditioning

The table is designed to require the minimum maintenance. It is recommended that preventive maintenance be carried out according to our service schedule every other year.

### **Normal Cleaning**

Mechanics cleaned with a damp cloth. Readily available cleansers and disinfectants can be used. Can be rinsed with water.  
The table base and eating/reading board are cleaned with a damp cloth. Readily available cleansers and disinfectants can be used.

### **High-pressure Wash**

Before washing castors must be in locked position.

## 13- Warranty / Lifeage

**Warranty Time:** 2 Years

**Expected Lifeage:** 10 Years (In case used properly with maintenance program)

**Date:** 06.03.2021

**User Manual Number:** K058-PTS / R.210823

Please contact us for any questions:

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Human Care AB Årstaängsvägen 21B, 117 43 Stockholm/SVERIGE