PEM Einweg-Haltegürtel PEM Disposable Support Belt

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Bruksanvisning

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1 Allgemeine Informationen

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Table of Contents

1 General Information

- 1.1 Product versions
- 1.2 Accessories
- 1.3 Scope of delivery
- 1.4 Storage, transport
- 1.5 Service life / lifecycle

2 Safety information

- 2.1 Symbols used
- 2.2 Function check

3 Using the product

- 3.1 Indication and further use
- 3.2 Contraindications
- 3.3 Application risks
- 3.4 Areas of application
- 3.5 Application illustration
- 3.6 Application
- 3.7 Approved accessories, combination with other products

4 Product marking / CE

5 Cleaning, care and disinfection

6 Technical Data

7 Warranty

8 Disposal information

1 General Information

Thank you for choosing a Petermann product.

Please read the user manual carefully before using the product for the first time!

The current understanding in modern occupational health and safety is that people in nursing professions experience significant work-related physical strain. Back injuries are common. There are many situations in which a patient has to be transferred from a bed, a wheeled stretcher, ambulance stretcher, an X-Ray table or a wheelchair to a different stretcher or seat. The strain on the user and the patient / care-dependent can be reduced significantly through the use of sliding mats.

The use of transfer aids should be conditional upon the user having learned to use the equipment properly. The user has to decide for themselves what equipment is suitable to the patient's situation and the intended sequence of movements.

The Disposable Support Belt is part of a product range with a wide range of possible applications. This product is used in particular when transferring seated patients in hospitals and other health care facilities as well as home care settings.

The product may only be used for the specified purpose and under the specified conditions.

The Disposable Support Belt enables professional support of the patient without hygienic risks.

The information in this user manual is for the patient who is being transferred and moved, as well as the user performing the transfer / position change.

Contact us if you require additional information or assistance using the Petermann product.

1.1 Product versions

Item no.: PM-070.10-5 Disposable Support Belt (5 pieces)

1.2 Accessories

No accessories are available for this product.

1.3 Scope of delivery

Disposable Support Belt (5 pieces)
User manual

1.4 Storage, transport

Store/transport at -5°C – 40°C
Other: protect from sunlight

1.5 Service life / lifecycle

The Disposable Support Belt is a disposable product and may only be used for one person. It is not suitable for processing or reuse.

2 Safety information

WARNING indicates a potential danger which could lead to personal injury or material damage.

- It is the responsibility of the organisation / the user to determine, on the basis of the circumstances (weight / health condition of the patient) and the capabilities of the user, whether several users may be required for the transfer and/or whether additional safety measures are required.
- Defective products may no longer be used.
- Risk of accident! When using the Disposable Support Belt always ensure that the bed, the stretcher or any sort of mobile support whatsoever is standing securely, i.e. that brakes on it are engaged.
- It is essential to ensure that the number of users is sufficient to ensure a safe transfer in the given situation.
- Ensure that the transfers are performed by the users in a back-friendly manner.
- Weight limits must be observed (160 kg).
- Ensure that the Velcro fastener on the belt closes securely during use and verify that the Velcro material is working properly. A belt with defective Velcro material must not be used any longer.
- Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.
- The Disposable Support Belt must not be reprocessed. Reprocessing of the foil can lead to material damage and thus result in harm to the patient.
- Apply the Disposable Support Belt tightly so that a safe transfer is possible and make sure that the patient is not constricted. Remove the product after the transfer.
- Pay attention to possible risk of tripping and getting caught in the belt systems!

2.1 Symbols used

	For single use only		max. load capacity see item 6
	Warning		Manufacturer
	Observe the user manual		Batch no. / lot no.
	Conformity mark		Identification code
	Item number		Medical device label

2.2 Function check

The Disposable Support Belt is a single-use product and may only be used on one person at a time. It is not intended for reprocessing or reuse.

➤ **Defective products may no longer be used!**

3 Using the product

3.1 Indication and further use

Due to the wide range of possible applications, the indications cannot be tied to a specific clinical picture, since . Support Belts are suitable for all applications involving people who require assistance.

- Conveying security when changing position or walking
- Muscle tension can be reduced
- Increase in independence

3.2 Contraindications

Contraindications cannot be tied to a specific clinical picture. Under consideration of the safety information (see item 2), the product may not be used if the patient has any of the following conditions:

- Fractures of the lower extremities
- Multiple paralysis (only in combination with additional products)

3.3 Application risks

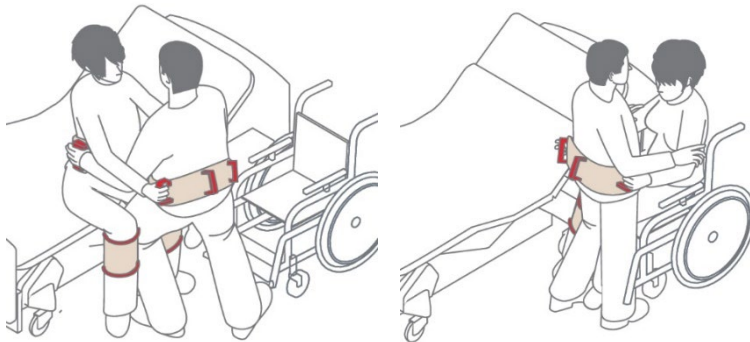
- Improper fastening → risk of falling
- Material damage and the resulting risk of falling

3.4 Areas of application

The product can be used in other health care facilities as well as hospitals.

3.5 Application illustration

3.6 Application

Anyone who takes on the professional care of a person is constantly challenged anew, especially with regard to compliance with the regulations for disinfection and hygiene of the aids used.

The specially developed Disposable Support Belt gives the caregiver a new kind of transfer and ultimately care safety.

The Support Belt is placed around the hips or waist of the patient, closed with a size adjustable quick fastener and pulled tight.

The Disposable Support Belt can be individually adjusted to the correct length for each patient using scissors.

3.6.1 Transferring while sitting

Prerequisites for the transfer

- adequate trunk stability
- sufficient muscle tone in the legs

Option A

- Grasp the belt
- Step forward
- Shift weight backwards
- Turn in the desired direction

Option B

- Assistance with kinaesthetic seated transfers
- Combination with the sliding board

3.6.2 Transfer while standing

Option A

- Grasp the belt
- Step forward
- Shift weight to the back while providing assistance to stand up
- Use small steps to steer in the desired direction
- Step forward
- Tense
- Assist with safe sitting down

Option B

- Mobilisation to standing (see option A)
- Assist with safe walking

3.6.3 Other

- The user can also put on a Support Belt to facilitate a safer and more fluid transfer
- Anxious patients are given a sense of safety

Visit our website at www.pm-med.de or watch our comprehensive product videos on our YouTube channel.

3.7 Approved accessories, combination with other products

Disposable Support Belts are used in combination with the following products:

- PM-075-10 Disposable Leg Belt
- PM-5080 Turning Disc
- PM-5020/30/50 Sliding Board

Petermann transfer aids are part of a comprehensive aid product range, which are effective and back-friendly for the user and comfortable and supportive for the patients. Additional Petermann products may be used according to the situation and clinical picture. Ask our experts. We are happy to help you!

4 Product marking / CE

The Petermann Disposable Support Belt is a Class I Medical Device and meets the EU-Regulation 2017/745 of the European Parliament and Council dated 05.04.2017 for medical devices, also called MDR.

Petermann GmbH is certified in accordance with DIN EN ISO 13485 by TÜV Süd, Medical Devices - Quality management systems applicable to the design and development, production and distribution of aids for people of limited mobility.

5 Cleaning, care and disinfection

The Disposable Support Belt is a single-use product and may only be used on one person at a time. It is not intended for reprocessing or reuse.

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The manufacturing of our products is subject to highest quality standards. Due to usual manufacturing variations published dimensions and further information concerning the condition of the product are defined as target production specifications. Deviations which might occur are monitored by professional quality management procedures and do not represent a reduction in performance of the respective product.

7 Warranty

- The company Petermann warrants that the product is free from defects in material and workmanship.
- The company Petermann is not liable for accidental or consequential injury or material damage.

8 Disposal information

There are no specific disposal regulations for Disposable Support Belts, they can be disposed of in accordance with applicable national and local regulations.

Petermann GmbH
Waldweg 3
D-91601 Dombühl
Phone: +49 (0) 9868-9339-0
Email: petermann@pm-med.de

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