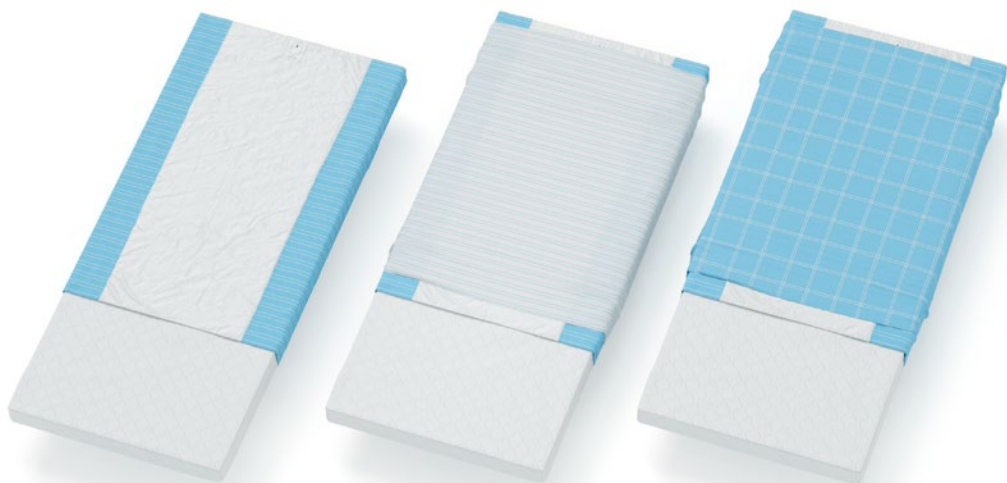
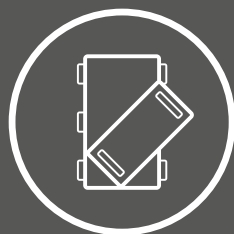
# HUMAN CARE

Your Life. Your Way.

Alpha Base Sheet 2-way /  
Alpha Draw Sheet 2-way /  
Alpha Draw Sheet 4-way

User manual / Bruksanvisning

## Content

### ENGLISH

User Manual	3
Compliances and Standards	4
Safety Instructions	4
Intended use	4
Technical Specifications	5
Product Label	8
Application and Use	10
Care instructions	13

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# User Manual

## Important!

You must read the user manual of your device prior to usage. Keep this booklet and information handy for future use.

## Symbols

**WARNING** symbol indicates a potentially hazardous situation which, if not avoided, could result in serious injury to the user or damage to property and/or the device itself.

**INFORMATION** symbol indicates recommendations and information for proper, trouble-free usage.

**WEIGHT CAPACITY** symbol indicates maximum user weight for the product. Do not exceed this weight in any circumstance.

**CE MARKING** symbol indicates that the product meets requirements of EU Regulation 2017/745 (MDR).

**MEDICAL DEVICE** symbol indicates that the product is a medical device according to the definition of medical device in EU Regulation 2017/745 (MDR).

**RECYCLE** according to local regulations.

**READ INSTRUCTIONS** and ensure you fully understand them before using this product.

***Product modification is not permitted. Before use, always check that the product is not damaged or worn.***

Human Care's products are continuously being developed and updated to provide our customers with the highest quality. We reserve the right to make design changes without prior notice.

Always make sure that you have the most recent version of the manual which is available for downloading from our website at [www.humancaregroup.com](http://www.humancaregroup.com).

Contact your local distributor if you have any questions about the product and its use.

### Notice to user/patient in case of serious incident

Any serious incident that has occurred in relation to the product, should be reported to the local contact, who reports to the manufacturer, and the competent authority of the country in which the user/patient is established.

# Alpha Sliding Sheets

## Compliances and Standards

---

Human Care is an ISO 13485:2016 certified medical device manufacturer. The Quality Management System complies with US 21 CFR

Part 820. The product is CE-marked as a Class I medical device under EU MDR 2017/745.

## Safety Instructions

---

The process of transferring and moving a person always involves risks. For the safety of the client and the care givers, you must read the user manuals for the sheets and the related equipment carefully and conduct a risk assessment.

Check that the right product for the client is being used. The model, size and material must meet the client's needs.

Wash all sheets before first use with a client.

Carry out a careful visual inspection of the product every time you use it. Ensure that the product is in sound condition and that the material and the seams are not damaged.

Make sure no sharp objects are in contact with the product. Defective products may no longer be used.

Check the transfer surfaces carefully to ensure they are fixed in place securely (e.g. brakes of bed are activated).

Never leave sheets on the floor.

Be aware that there is a gliding risk when sitting on the edge of the mattress.

After use, the gliding movement can be inhibited by bedding the Draw Sheet in. The hanging parts should always be folded under the mattress on both sides of the bed.

## Intended use

---

The Sliding Sheets reduce the friction at pressure points when positioning a client in bed. The client has limited abilities to perform these movements and/or need help of a care giver for whom it can be a heavy load on shoulders and back to position the person without using devices. The client can stay on these sheets. No additional sheets are required.

### **Alpha Base Sheet 2-way:**

The sliding surface of the Based sheet is finished with extremely low friction sliding material. This facilitates positioning and turning in bed by the client. The shorter versions are for active clients, who can be able to place their feet on the mattress. The full-length version is for less mobile clients, who get support of a

care giver. The fitted version fits also well on air-mattresses. For clients with a low risk of incontinence during bed rest, some Alpha Base Sheets are available in waterproof satin.

### Alpha Draw sheet 2-way:

The Draw Sheet 2-way allows movement in sideward direction for manual turning or positioning in the bed from/towards the side edge with help of a care giver. It prevents the client from gliding down in bed.

### Alpha Draw Sheet 4-way:

The Draw Sheet 4-way allows movement in sideward and upward direction, for positioning

higher up in bed with lesser friction. It is intended for clients who cannot assist and/or for heavy clients. Clients have possibility to be more active during the transfer. The Draw Sheet 4-way are for manual positioning. There are some models with handles, they may be used with a lift together with a 2-point hanger bar. Draw Sheet 4-way with handles is also available in 'inco' version that provide strong absorption for higher protection needs.

These products may be reused.

Before reusing the product, be sure to perform a thorough function check as well as cleaning or disinfection of the product.

The product can be used in home care and other health care facilities as well as hospitals.

Alpha Draw Sheet 2-way

Alpha Draw Sheet 4-way

Alpha Draw Sheet 4-way  
Handles

## Technical Specifications

The Base Sheet 2-way and Draw Sheet 4-way have a sliding surface, made from a satin

polyester, and a printed terylene/cotton (polyester) to present the sliding directions.

Alpha Base Sheet 2-way

Alpha Draw Sheet 2-way

Alpha Draw Sheet 4-way

# Alpha Sliding Sheets

## Alpha Base Sheets 2-way

Item no	Product name	Dimensions – product (length x width)	Dimensions-sliding surface (length x width)	Fits bed width	Material	Color	Pattern
PM-1611	Alpha Base Sheet 2-way 100x200 cm	100x200 cm	100x50 cm	80 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1629	Alpha Base Sheet 2-way 100x200 cm	100x200 cm	100x60 cm	80-90 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1616	Alpha Base Sheet 2-way 100x220 cm	100x220 cm	100x75 cm	105 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1667	Alpha Base Sheet 2-way 100x220 cm	100x220 cm	100x220 cm	120 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1612	Alpha Base Sheet 2-way 140x200 cm	140x200 cm	140x50 cm	80 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1634	Alpha Base Sheet 2-way 140x200 cm	140x200 cm	140x60 cm	80-90 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1621	Alpha Base Sheet 2-way 140x220 cm	140x220 cm	140x75 cm	105 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1668	Alpha Base Sheet 2-way 140x220 cm	140x220 cm	140x95 cm	120 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1613	Alpha Base Sheet 2-way 200x200 cm	200x200 cm	200x50 cm	80 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1639	Alpha Base Sheet 2-way 200x200 cm	200x200 cm	200x60 cm	80-90 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1622	Alpha Base Sheet 2-way 200x220	200x220 cm	200x75 cm	105 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1669	Alpha Base Sheet 200x220 cm	200x220 cm	200x95 cm	120 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1681	Alpha Base Sheet 2-way 200x280 cm	200x280 cm	200x60 cm	140-200 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1641	Alpha Base Sheet 2-way fitted 200x90 cm	200x90 cm	200x70 cm	80-90 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1651	Alpha Base Sheet 2-way fitted 200x105 cm	200x105 cm	200x80 cm	105 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1661	Alpha Base Sheet 2-way fitted 200x120 cm	200x120 cm	200x95 cm	120 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Striped
PM-1629W	Alpha Base Sheet 2-way waterproof 100x200 cm	100x200 cm	100x60 cm	80-90 cm	Cotton/terylene (C/T 55/45%) and terylene polyurethane	Blue/white	Striped
PM-1634W	Alpha Base Sheet 2-way waterproof 140x200 cm	140x200 cm	100x60 cm	80-90 cm	Cotton/terylene (C/T 55/45%) and terylene polyurethane	Blue/white	Striped

## Alpha Draw Sheets 2-way and Alpha Draw Sheets 4-way

Item no	Product name	Dimensions – product (length x width)	Fits bed width	Material	Color	Pattern
PM-1631	Alpha Draw Sheet 2-way 100x200 cm	100x200 cm	80-120 cm	Cotton/terylene (C/T 55/45%)	White/blue	Striped
PM-1636	Alpha Draw Sheet 2-way 140x200 cm	140x200 cm	80-120 cm	Cotton/terylene (C/T 55/45%)	White/blue	Striped
PM-1637	Alpha Draw Sheet 4-way 140x200 cm	140x200 cm	80-120 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Checked
PM-1647	Alpha Draw Sheet 4-way 200x200 cm	200x200 cm	80-120 cm	Cotton/terylene (C/T 55/45%) terylene	Blue/white	Checked
PM-1657	Alpha Draw Sheet 4-way Handles 140x200 cm	140x200 cm	80-120 cm	Cotton/terylene (C/T 55/45%) terylene, handles polypropylene	Blue/white	Checked
PM-1687	Alpha Draw Sheet 4-way Handles 200x200 cm	200x200 cm	80-120 cm	Cotton/terylene (C/T 55/45%) terylene, handles polypropylene	Blue/white	Checked
PM-1649	Alpha Draw Sheet 4-way Handles Inco 140x120 cm	140x120 cm	80-120 cm	Terylene polyester, viscose, polyurethane layer, terylene, handles polypropylene	Blue	No pattern

## Kits: Alpha Draw Sheet + Alpha Base Sheet 2-way

Item no	Item no Draw Sheet + Item no Base Sheet 2-way	Fit bed width
PM-1630	PM-1631+PM-1629	80-90 cm
PM-1642	PM-1636+PM-1641	80-90 cm
PM-1635	PM-1636+PM-1634	80-90 cm
PM-1640	PM-1636+PM-1639	80-90 cm
PM-1650	PM-1636+PM-1669	120 cm
PM-1638	PM-1637+PM-1634	80-90 cm
PM-1644	PM-1637+PM-1641	80-90 cm
PM-1643	PM-1637+PM-1639	80-90 cm
PM-1655	PM-1637+PM-1651	105 cm
PM-1645	PM-1647+PM-1641	80-90 cm
PM-1665	PM-1647+PM-1669	120 cm
PM-1628	PM-1634+PM-1657	80-90 cm
PM-1646	PM-1637+PM-1641	80-90 cm
PM-1656	PM-1657+PM-1651	105 cm
PM-1658	PM-1634+PM-1649	80-90 cm
PM-1699	PM-1649+PM-1639	80-90 cm

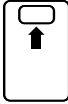
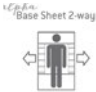
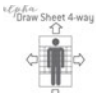
### Notes:

- The load-bearing capacity is < 200 kg/<440 lbs for each sheet.
- Discrepancies in measurements can occur, tolerances  $\pm 1$  cm, 0,3". The shrinkage percentage due to laundry is  $\pm 5\%$ .

# Alpha Sliding Sheets

## Product Label

Each sheet product has 3 labels:

Satin sheet item number	Illustration
Label 1: Location of bed head end	 <b>THIS SIDE UP</b>
Label 2A: Alpha Base Sheet 2-way sideways/ turning in bed	 Alpha Base Sheet 2-way
Label 2B: Alpha Draw Sheet, 2-way sideways/ turning in bed	 Alpha Draw Sheet 2-way
Label 2C: Alpha Draw Sheet 4-way side-/upwards and turning in bed	 Alpha Draw Sheet 4-way

Label 3 on the product contains the following information:

**Alpha Base Sheet 2-way**  
**140x200 cm**

Flat, sideways/turning in bed, bed width 80-90 cm,  
sliding surface length x width 140x60 cm

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**EAN**   
0 123456 789012

**UPC**   
0 123456 789012

	80°C/176°F laundry
	Do not bleach
	Dry at low temperatures (45° - 50°C/113-122°F)
	Iron at low temperature (<110°C/230°F)
	Do not chemical cleaning
	Latex free
	Do not use softener / rinsing agent
	Read the user manual
	Company logo
	GS1-128 barcode
	Unique Product Code, UPC barcode

	max. load capacity
	Manufacturer
	Identification code
	Conformity mark
	Item number
	Visual inspection
	Medical device label
	Serial number
	Country of Manufacturer, abbreviation country code
	Unique Product Code for North America

# Alpha Sliding Sheets

## Application and Use

General: Transfers shall always be performed in a back-friendly and ergonomic manner.

### Alpha Base Sheet 2-way

- Put the Base Sheet on the mattress like a normal sheet with all labels towards the head end of the bed. The sliding surface should be positioned centrally, under the main friction points, for example the shoulders and hips.
- Ensure the non-slip surfaces (blue/white fabric) are evenly positioned on both sides of the mattress. These surfaces prevent the client from sliding out of the bed.

Alpha Base Sheet 2-way  
100 cm length

Alpha Base Sheet 2-way  
140 cm length

Alpha Base Sheet 2-way  
200 cm length

### Alpha Base Sheet 2-way fitted

- Fit the Base Sheet onto the mattress by using the elastic band. The sliding surface is facing up and the label 'this side up' is visible towards the head end.
- The opening on the corners may be used to route the air mattress tubes.

### Alpha Draw Sheet 2-way and 4-way

Place the Draw Sheet directly onto the Base Sheet.

The printed side of the Draw Sheet facing up.

Alpha Draw Sheet 2-way

Alpha Draw Sheet 4-way

## Turning with a Alpha Draw Sheet 2-way and 4-way

- Grip the fabric of the Draw Sheet at the level of the shoulders and at the level of the hips. Roll the fabric to improve grip.
- Keep wrists straight and shift body weight by leaning backwards.
- Turning may be performed by one or two care givers. This asks for cooperation: One care giver pulls the Draw Sheet as described above. The other person pulls down the Draw Sheet, close to the shoulder and hips at the other side of the client's body.
- The client turns.
- Always tuck the draw sheet under the mattress on both sides to prevent gliding.
- Smooth out bedding before tucking to reduce the risk of chafing.

***Every time you use a Draw Sheet 4-way you should have the bedrails up after use!***

Alpha Draw Sheet 2-way

Alpha Draw Sheet 4-way

## Higher up in bed with Alpha Draw Sheet 4-way

- Bring the mattress in a horizontal position.
- Grasp the Draw Sheet on both sides near the shoulders.
- Stand in a walking position and step backwards.
- To prevent the user from sliding down, fold part of the Draw Sheet double under the client's seat.

## Sitting up in bed with Alpha Draw Sheet 4-way

- Fold the draw sheet under the client's seat and tuck the draw sheet under the mattress on both sides. This prevents the user from gliding down when raising the head end of the bed.
- When the client is in the correct position, tuck in the whole draw sheet under the mattress. The Draw Sheet acts as a friction surface.

# Alpha Sliding Sheets

## Turning with assistance of a lift with Alpha Draw Sheet 4-way Handles

- Select two of the handles, which are close to the friction points (shoulder and hips) on one side and attach them to the hooks of the hanger bar.
- Prepare the client by placing the body in a start position for turning.
- Start turning by raising the hanger bar carefully in small steps with regular pauses to make any necessary adjustments, e.g., move the client closer to the middle of the bed to free more space for turning.
- After turning, detach the handles from the hanger bar.

Alpha Draw Sheet 4-way Handles

Attached Alpha Draw Sheet 4-way Handles to hangerbar

## Hygiene activities with assistance of a lift and Alpha Draw Sheet 4-way

Pull the lower edge of the Draw Sheet in so that it is folded double. The lower body part of the client is now approachable. The hanger bar can

stay attached to the handles for supporting the client in the sideward position.

## Prone positioning with Alpha Draw Sheet 4-way Handles

### Procedure

#### 1. Initial Positioning

If the client is connected to a ventilator or other life support equipment, turn them towards the equipment.

Place the client's arm on the turning side under their hip.

If required, place 2–3 pillows or positioning cushions on top of the client.

#### 2. Setting up Draw Sheet (B)

Place a second Draw Sheet 4 Way with Handles (B) on top of the pillows, satin side up.

Fold approx. 20 cm of Draw Sheet (B) away

from the client's head.

Tuck Draw Sheet (B) snugly around the client, creating a cocoon.

#### 3. Preparing for the Turn

Move the client slightly sideways to prepare for rotation.

Fold Draw Sheet (A) away from the head so satin faces satin.

Assign roles:

One caregiver supports and stabilizes the head.

One caregiver on each side of the bed assists with turning.

#### 4. Turning the Client

Attach the handles of Draw Sheet (A) to the hanger bar hooks.

Slowly turn the client to nearly 90° (side lying).

Gently push and pull on Draw Sheet (A) to position the client slightly off center.

While the client is side lying:

One caregiver tucks Draw Sheet (A) under the client for easy removal later.

The same caregiver supports the shoulder.

Lower the hanger bar.

Remove the handles from the hooks and remove Draw Sheet (A).

Unfold Draw Sheet (B) under the client's head.

Reposition the client to the center of the bed if needed.

Ensure proper alignment and comfort.

Raise bed rails if required.

#### 5. Moving to Prone

Continue turning the client until they are fully prone.

#### Returning to Supine

To return the client to a supine position, follow the same steps in reverse order.

## Care instructions

### Check for wear and tear

Check the product for wear and tear. Inspect the fabric, seams and handles regularly.

Do not use damaged products.

Transfer products are subject to a natural ageing process. Inspect the equipment regularly to ensure proper functionality.

### Cleaning

Human Care sheets are intended for one patient at the time. To avoid infections or transmissions, it is important to clean the sheets properly. All sheets can be machine washed at 80°C and hang dry or tumble dry at low temperatures (45° - 50°C). We recommend washing with mild detergents.

Sheets should always be cleaned after use in a wet situation. This product can be wiped with 70% disinfection solution. Do not use bleaching agents, fabric softeners, or other non-standard cleaning agents, as these may impair sliding properties.

Always follow the care instructions on the label of every product as well as the information in the respective user manual.

***Always refer to the product label before washing.***

### Store/transport

Store and transport the product within a temperature range of -30°C and + 40°C. The product should be protected from UV/sunlight.

# Alpha Sliding Sheets

## Periodic inspection

Check the product functionality and ensure that the material withstands the load, especially for the models with handles. The smooth surfaces significantly reduce resistance due to friction during repositioning. Check these properties - ideally in comparison to a new product.

### Steps:

1. Place the product on a firm surface
2. Pull your hands over the product with pressure.

Evaluation of result:

- o Low friction - easy gliding
- o High friction: no gliding/moving possible

## Product lifetime

The expected product lifetime one or two years, if the product is used as intended and maintained according to the manufacturer's

instructions, depending on the intensity of use, laundry process and maximum load applied during use. If the printed product label is no longer legible, the product should be discarded. It does not cover defects attributable to abrasion, normal wear, improper use or improper cleaning.

## Warranty and support

If you need information or support, please contact your Human Care office, [www.humancaregroup.com](http://www.humancaregroup.com), or distributor.

## Recycling

Once the product has reached the end of its life, it should be disposed according to your country's rules, especially for the separate collection of textile products. Correct disposal helps prevent negative consequences for the environment and human health.

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